

May 28, 2004

Motion Concepts

ON A ROLL
A weekly e-bulletin from Motion Concepts

Back to the future

The future of superior back support and comfort has never been more clear ... thanks to the new **MaTRx-PB (Posture Back)** from Motion Concepts.

Available June 1, 2004, the MaTRx-PB is designed to provide extraordinary comfort and postural support for the widest range of user needs — from highly active clients who need mainly low back and hip support, to more involved clients looking for a higher degree of trunk support.

The MaTRx-PB is available in 3 back widths 16", 18" and 20" and 3 heights: short (12"), regular (16") and tall (20"). It features adjustable mounting pins that enhance flexibility by allowing each back width to fit 3 different wheelchair widths. For example, the 16" wide backs fit into wheelchairs that are 15", 16" or 17" wide; the 18" wide backs fit into 17", 18" or 19" wheelchairs and the 20" backs fit into 19" and 20" wheelchairs. All this means users with wider hips and narrower trunks (or the reverse) get an optimal fit in their chairs.

Our short back is ideally suited to highly active users who need added support for their hips and lower backs, and would also like to be able to fine tune their centers of gravity within their chair. The regular and tall back heights accommodate clients with more complex needs because they provide a higher level of trunk support.

All models are extremely lightweight at approximately 3.5 lbs, including hardware and feature quick release hardware for easy portability. They also feature innovative "Easy Set" hardware, which has a multitude of benefits, including easy installation and adjustment in just minutes! This hardware also provides infinite adjustment of recline (up to 30°) and seat depth (up to 4") and **even accommodates rotation!** Best of all, every adjustment can be made while the client is seated in their wheelchair so there's no need to transfer in and out while the back is adjusted!

The MaTRx-PB Back is also available in a Heavy Duty version, which is specially created to fit 20" - 25" and handle a 500 lb weight capacity. Extended Shear Reduction interface mounting for the MaTRx-PB is also available for use with Motion Concepts TRx Power recline systems.

So, help your clients experience the future of back support and comfort with the revolutionary, new MaTRx-PB from Motion Concepts. From the most active to the more involved — they'll be glad you did.



MaTRx-PB short back for highly active users



MaTRx-PB reg/tall back for less active users

www.motionconcepts.com
U.S.
700 Ensminger Rd.,
Suite 112
Tonawanda, NY 14150
TEL 1.888.433.6818
FAX 1.888.433.6834

Canada
84 Citation Dr.,
Concord, ON
L4K 3C1
TEL 905.695.0134
FAX 905.695.0138



"Easy Set" Hardware



Loaded Contour
Depth 5"