


# TRC0543 - KNEE ADDUCTOR ASS'Y, RIGHT

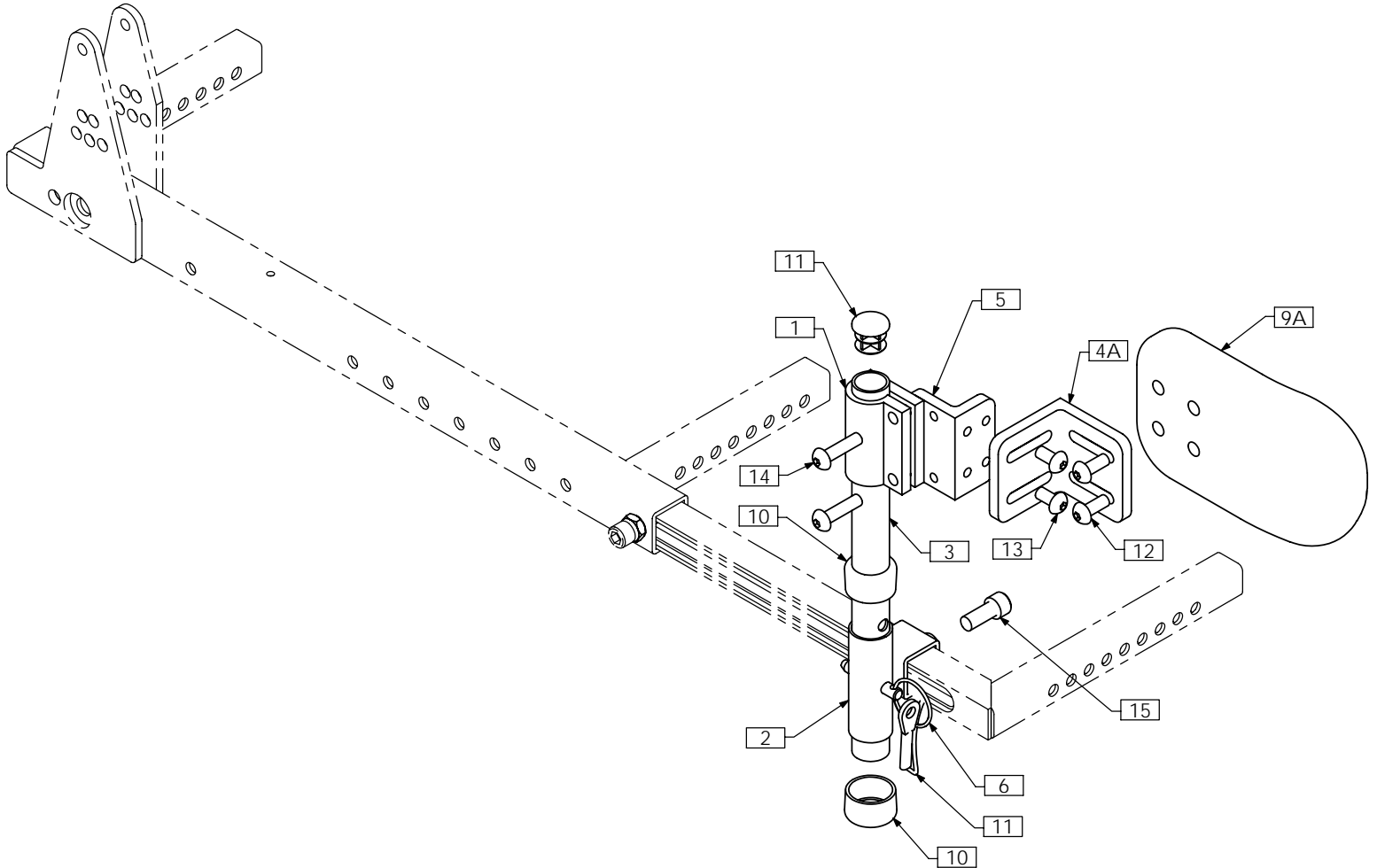

NOTES:  
1. N/S = NOT SHOWN

ITEM NO.	QTY.	PART NO.	DESCRIPTION	ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX0762	CLAMPING BRKT	9B N/S	1	TRM0337	PAD, MEDIUM
2	1	TRX3010	RECEIVER WLD	9C N/S	1	TRM0338	PAD, LARGE
3	1	TRX3013	ADDUCTOR POST	9D N/S	1	TRM0542	PAD, EXTRA SMALL
4A	1	TRX2273	ADAPTER BRKT, SLOTTED	10	2	TRM0431	REDUCER CAP, 7/8" - 3/4"
4B N/S	1	TRX2413	4 INCH "L" BRACKET	11	1	TRM0561	3/4" ROUND FLAT ROD CAP
5	1	TRX3384	ADAPTER BRKT, 1/4-20	12	2	TRH0308	1/4-20 x 5/8" BUTTON SOCKET CAP SCREW
6	1	TRM0103	PULL PIN	13	2	TRH0429	1/4-20 x 1/2" BUTTON SOCKET CAP SCREW
7	1	TRM0231	NYLON LANYARD	14	2	TRH0450	1/4-20 x 1" BUTTON SOCKET CAP SCREW
8 N/S	1	TRM0308	NEOPRENE SLEEVE	15	1	TRH0517	5/16-24 x 3/4" SOCKET HEAD CAP SCREW GR8
9A	1	TRM0336	PAD, SMALL				