


TRB0354 - HEADREST ASS'Y, ONYX SYSTEM

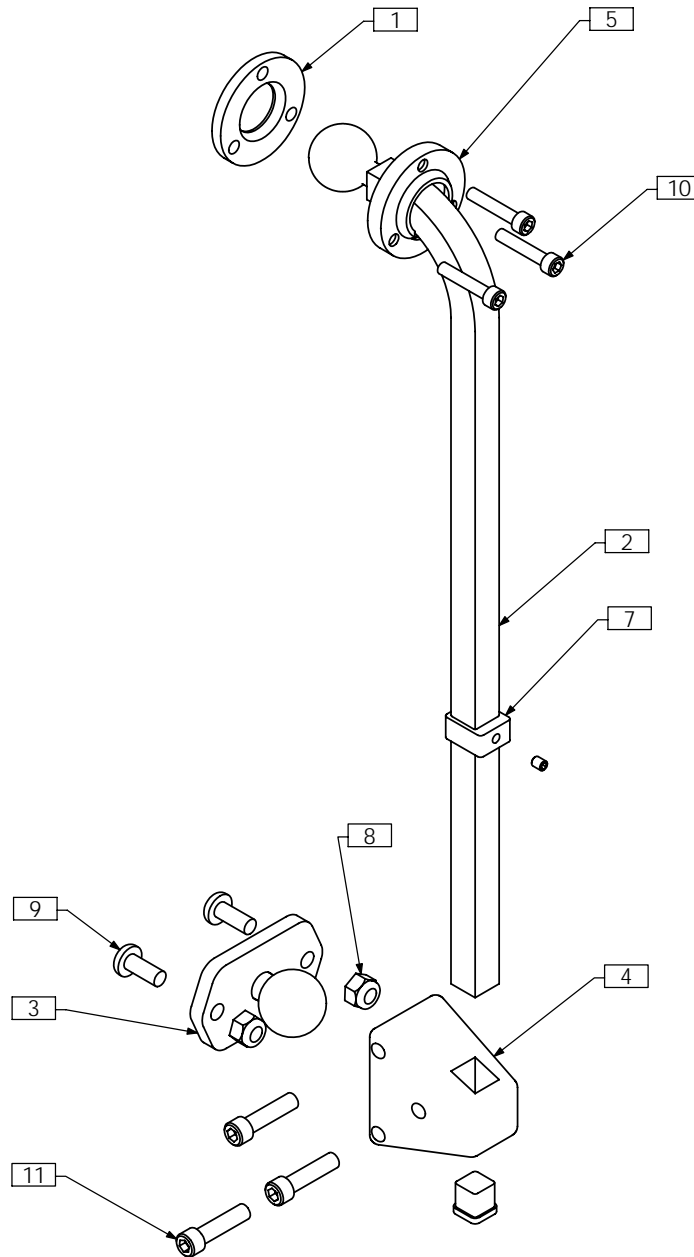

NOTES:
1. N/S = NOT SHOWN

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX1541	MOUNTING PLATE
2	1	TRX1567	TRX HEADREST MNT WLD
3	1	TRX1617	TRX PIVOT BLOCK MNT WLD
4	1	TRM0039	TRIANGULAR MULTI-OFFSET
5	1	TRM0040	HEADREST MOUNTING RING
6 N/S	1	TRM0368	ONYX HEADREST
7	1	TRM0414	D-RING
8	2	TRH0201	1/4-20 NYLOC NUT
9	2	TRH0309	1/4-20 x 5/8" PHILLIPS TRUSS MACHINE SCREW
10	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8
11	3	TRH0335	1/4-20 x 1" SOCKET HEAD CAP SCREW GR8