


TRC0435 - HEADREST ASS'Y, BASIC

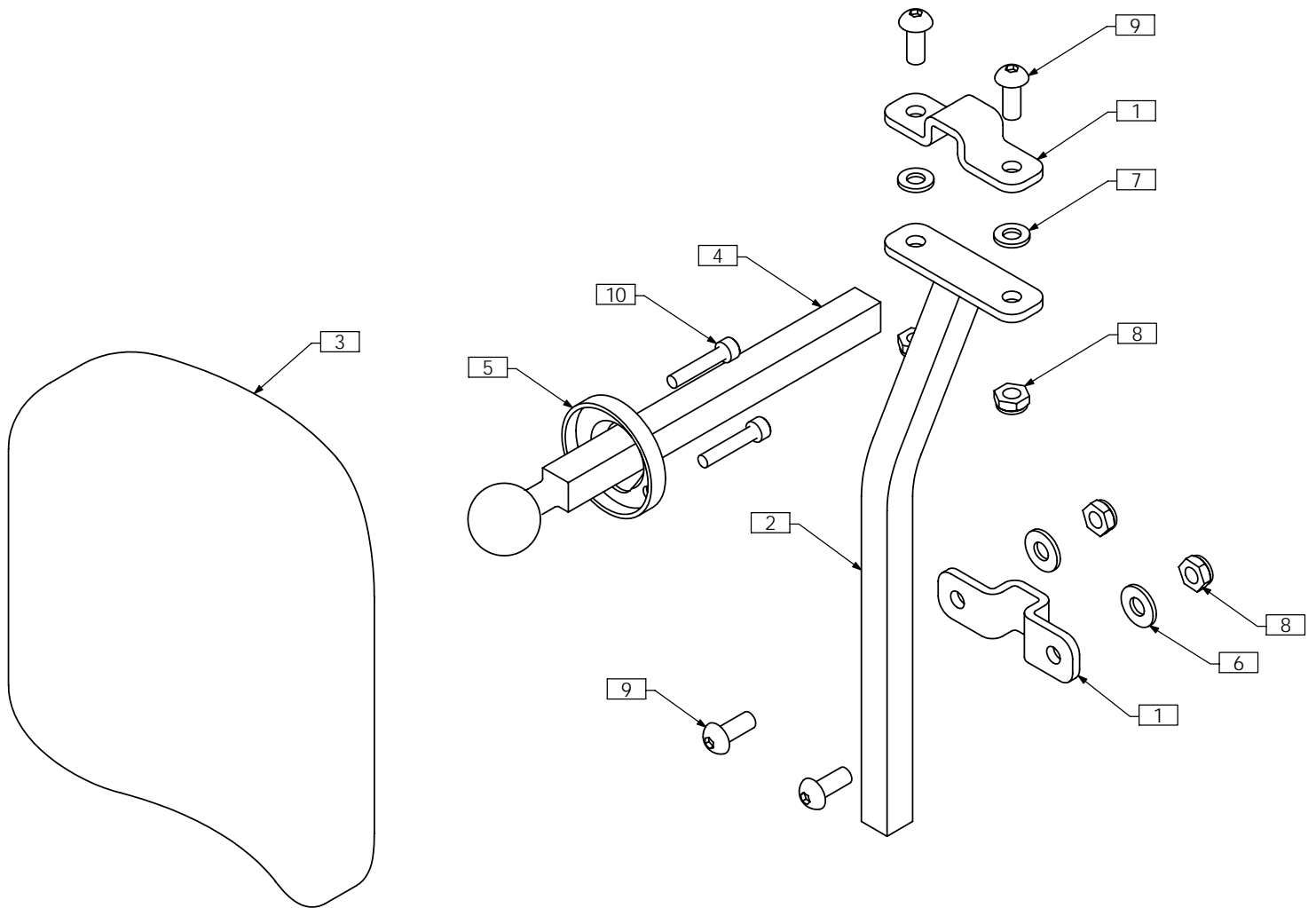

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	2	TRX2539	CLAMP
2	1	TRX2540	MOUNT WLD
3	1	TRM0005	HEADREST
4	1	TRM0037	SMALL HORIZONTAL HEADREST ROD
5	1	TRM0040	HEADREST MOUNTING RING
6	2	TRH0101	¼" FLAT WASHER
7	2	TRH0102	¼" NYLON WASHER
8	4	TRH0202	¼-20 SHALLOW NYLOC NUT
9	4	TRH0308	¼-20 x 5/8" BUTTON SOCKET CAP SCREW
10	3	TRH0329	#10-32 x 1" SOCKET HEAD CAP SCREW GR8