


TRC0470 - MANUAL ELEVATING DOWNTUBE ASS'Y, LEFT

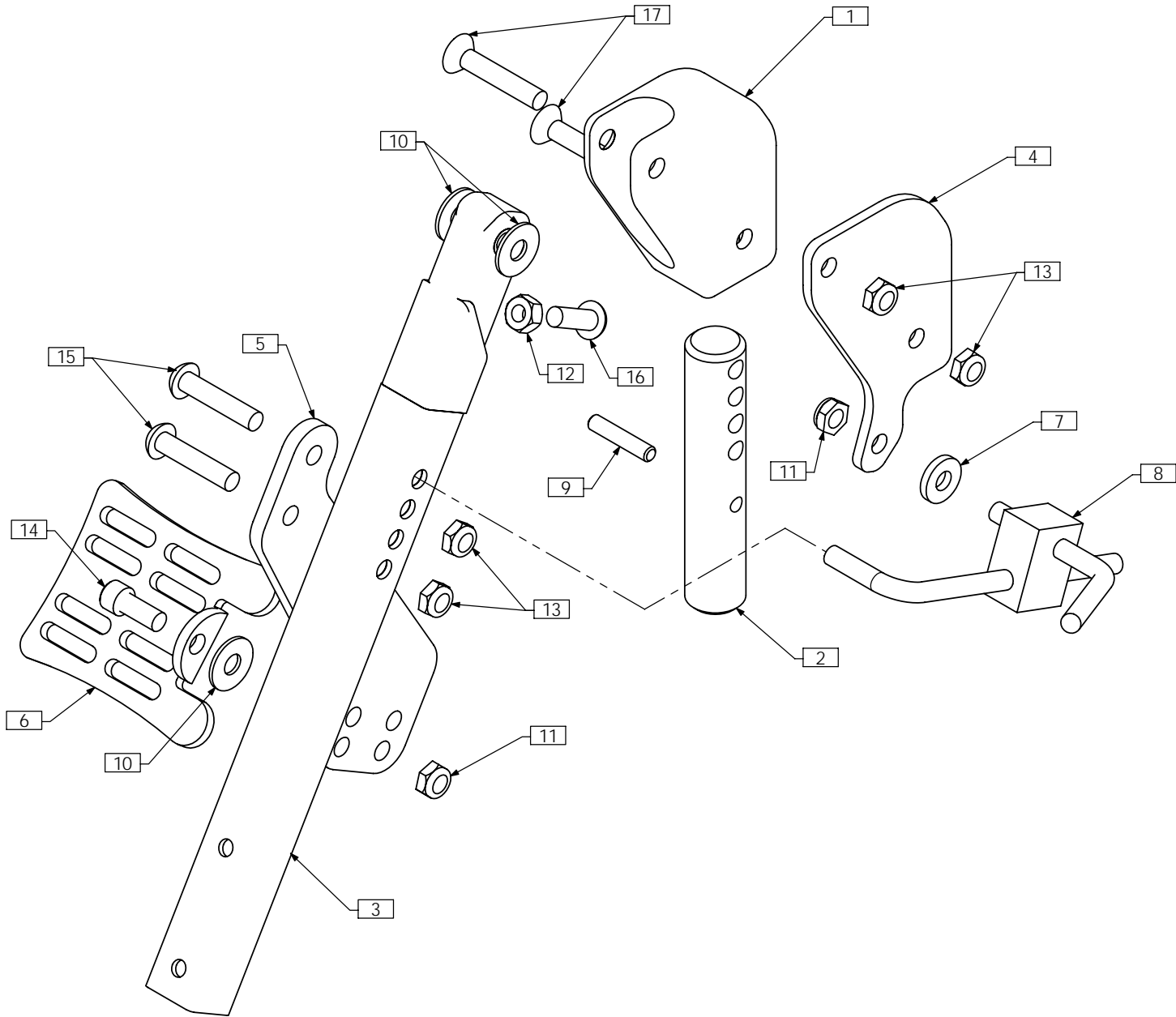

ITEM NO.	QTY.	PART NO.	DESCRIPTION	ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRX1395	PIVOT BLOCK, LEFT	10	3	TRH0109	5/16" NYLON WASHER
2	1	TRX0697	PIN	11	2	TRH0207	5/16-18 SHALLOW NYLOC NUT
3	1	TRX1155	DOWN TUBE ASS'Y	12	1	TRH0212	5/16-18 HEX NUT
4	1	TRX1264	ADAPTER PLATE	13	4	TRH0230	5/16-24 SHALLOW NYLOC NUT
5	1	TRX1265	CALF PAD MOUNTING BRACKET	14	1	TRH0323	5/16-18 x 3/4" SOCKET HEAD CAP SCREW GR8
6	1	TRX3198	CALF PAD MOUNTING BRKT	15	2	TRH0391	5/16-24 x 1 1/2" BUTTON SOCKET CAP SCREW
7	1	TRM0378	SPACER	16	1	TRH0430	5/16-18 x 7/8" BUTTON SOCKET CAP SCREW
8	1	TRB0367	CAM LOCK KIT	17	2	TRH0494	5/16-24 x 1 3/4" FLAT SOCKET CAP SCREW
9	1	TRH0022	DOWEL PIN, STAINLESS STEEL				