


# TRC0614 - MAIN ASS'Y, 3G STORM, SL/SP, RECLINE

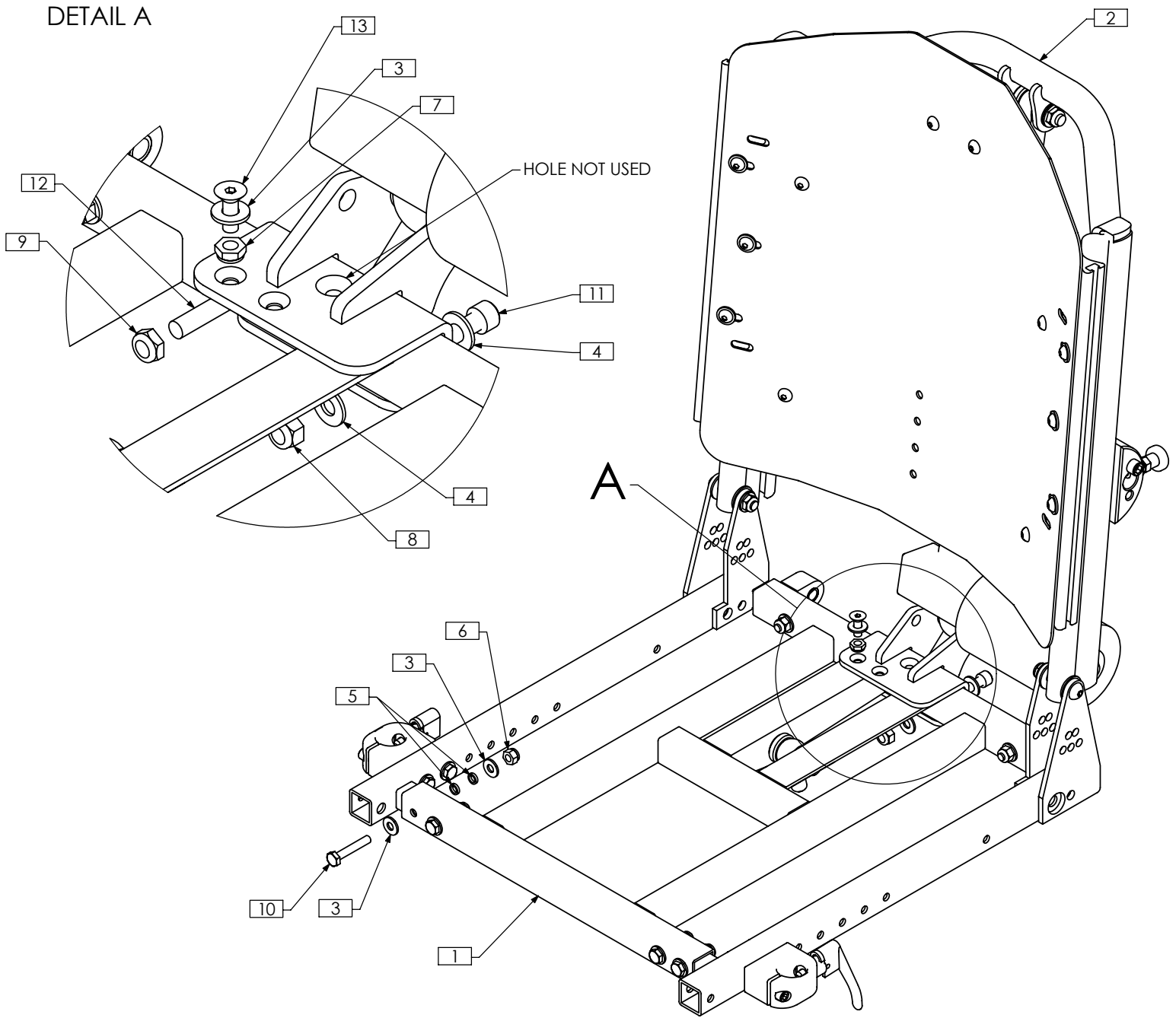

NOTES:  
 1. N/S = NOT SHOWN  
 2. SEAT PAN NOT SHOWN

ITEM NO.	QTY.	PART NO.	DESCRIPTION	ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC0612	SEAT FRAME ASS'Y, 3G STORM	8	1	TRH0206	5/16-18 NYLOC NUT
2	1	TRC0513	UPPER SEAT ASS'Y, RECLINE	9	1	TRH0207	5/16-18 SHALLOW NYLOC NUT
3	13	TRH0101	1/4" FLAT WASHER	10	6	TRH0301	1/4-20 x 1 1/2" HEX HEAD CAP SCREW
4	3	TRH0107	5/16" FLAT WASHER	11	1	TRH0315	5/16-18 x 1 3/4" SOCKET HEAD CAP SCREW GR8
5	12	TRH0113	1/4" HI COLLAR LOCK WASHER	12	1	TRH0332	5/16-18 x 1 1/2" SOCKET HEAD CAP SCREW GR8
6	6	TRH0201	1/4-20 NYLOC NUT	13	1	TRH0409	1/4-20 x 3/4" FLAT SOCKET CAP SCREW
7	1	TRH0202	1/4-20 SHALLOW NYLOC NUT				