


TRC0610 - MAIN ASS'Y, SL/SP, RECLINE

DETAIL A

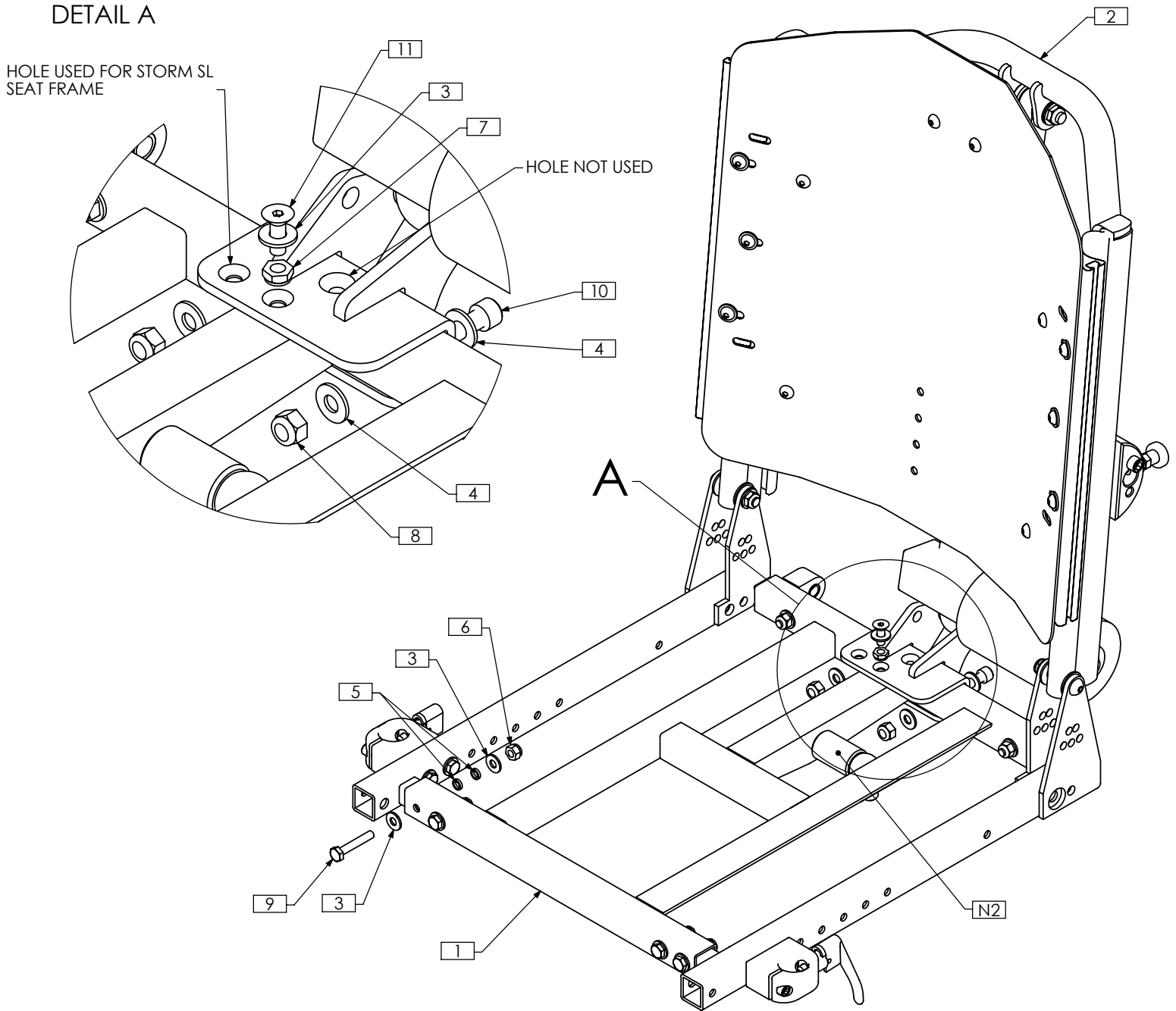

- NOTES:
 1. N/S = NOT SHOWN
 2. REFER TO TRC0189 FOR SPECIFIC BASE ROD END/SPACER CONFIGURATIONS AND ROD END HARDWARE
 3. SEAT PAN NOT SHOWN

ITEM NO.	QTY.	PART NO.	DESCRIPTION	ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC0189	SEAT FRAME ASS'Y	7	1	TRH0202	1/4-20 SHALLOW NYLOC NUT
2	1	TRC0513	UPPER SEAT ASS'Y, RECLINE	8	2	TRH0206	5/16-18 NYLOC NUT
3	13	TRH0101	1/4" FLAT WASHER	9	6	TRH0301	1/4-20 x 1 1/2" HEX HEAD CAP SCREW
4	4	TRH0107	5/16" FLAT WASHER	10	2	TRH0315	5/16-18 x 1 3/4" SOCKET HEAD CAP SCREW GR8
5	12	TRH0113	1/4" HI COLLAR LOCK WASHER	11	1	TRH0409	1/4-20 x 3/4" FLAT SOCKET CAP SCREW
6	6	TRH0201	1/4-20 NYLOC NUT				