


TRI134 - INSTALLATION OVERVIEW, ELITE POSTURE BACK ASS'Y, TILT-ONLY

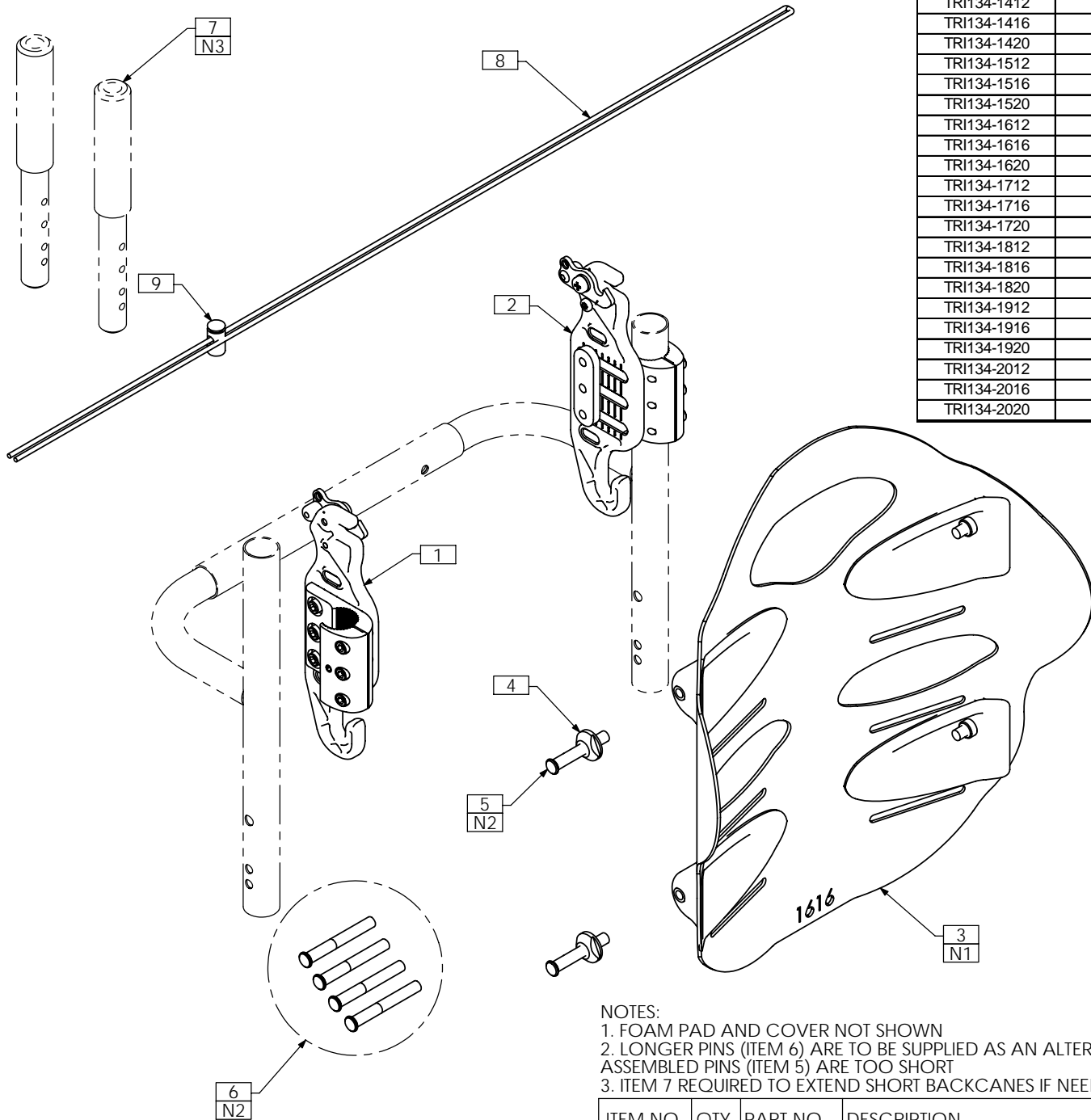

PART NUMBER	SIZE CONFIGURATION
TRI134-1212	12W X 12H
TRI134-1216	12W X 16H
TRI134-1312	13W X 12H
TRI134-1316	13W X 16H
TRI134-1412	14W X 12H
TRI134-1416	14W X 16H
TRI134-1420	14W X 20H
TRI134-1512	15W X 12H
TRI134-1516	15W X 16H
TRI134-1520	15W X 20H
TRI134-1612	16W X 12H
TRI134-1616	16W X 16H
TRI134-1620	16W X 20H
TRI134-1712	17W X 12H
TRI134-1716	17W X 16H
TRI134-1720	17W X 20H
TRI134-1812	18W X 12H
TRI134-1816	18W X 16H
TRI134-1820	18W X 20H
TRI134-1912	19W X 12H
TRI134-1916	19W X 16H
TRI134-1920	19W X 20H
TRI134-2012	20W X 12H
TRI134-2016	20W X 16H
TRI134-2020	20W X 20H

NOTES:

1. FOAM PAD AND COVER NOT SHOWN
2. LONGER PINS (ITEM 6) ARE TO BE SUPPLIED AS AN ALTERNATIVE IN CASE ASSEMBLED PINS (ITEM 5) ARE TOO SHORT
3. ITEM 7 REQUIRED TO EXTEND SHORT BACKCANES IF NEEDED

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	TRC0423	EASY SET MOUNTING ASS'Y, RIGHT, TILT-ONLY
2	1	TRC0424	EASY SET MOUNTING ASS'Y, LEFT, TILT-ONLY
3	1	TRS0062	ELITE POSTURE BACK CUSHION ASS'Y
4	4	TRX2390	MOUNTING PIN FLANGE NUT
5	4	TRX2829	MOUNTING PIN, SHORT
6	4	TRX2389	MOUNTING PIN
7	2	TRX2810	BACK CANE EXTENSION TUBE
8	1	TRM0549	#4 BLACK NYLON CORD, 5ft LENGTH
9	1	TRM0550	5/32 OVAL CORD LOCK