

MOTION CONCEPTS

Airmesh Calf Strap



Package Contents:

- (1) 36" Airmesh Calf Strap
- (2) Velcro™ Anchoring Strips

Mounting Location Options:

The Sports Calf Panel will fit around 16" - 18" widths

- ◆ Behind the calf
- ◆ In front of the lower leg
- ◆ Above the knee



Installation & Fitting Instructions:

1. Determine the desired mounting location.
2. Wrap the Velcro™ Strips around each footrest hangar bracket tube where the Airmesh Calf Strap is to be located.
3. Wrap the flexible part of the Airmesh Calf Strap over each Velcro™ Strip (this prevents sliding), with the padded side facing the body.

Cleaning: Machine wash warm (90° F, 32° C) with mild detergent. Do not use bleach. Line dry.

Warranty: All Motion Concepts products carry an unconditional guarantee against defects in workmanship and materials. Items found to be defective will be repaired or replaced at the discretion of Motion Concepts.

84 Citation Drive, Units 1-7, Concord, Ontario L4K 3C1
Toll Free Tel: 1.866.748.7943 Fax: 905.695.0

700 Ensminger Rd. Suite 112, Tonowanda, New York 14150
Toll Free Tel: 1.888.433.6818 Toll Free Fax: 1.888.433.6834



MOTION CONCEPTS

Airmesh Calf Strap



Package Contents:

- (1) 36" Airmesh Calf Strap
- (2) Velcro™ Anchoring Strips

Mounting Location Options:

The Airmesh Calf Panel will fit around 16" - 18" widths

- ◆ Behind the calf
- ◆ In front of the lower leg
- ◆ Above the knee



Installation & Fitting Instructions:

1. Determine the desired mounting location.
2. Wrap the Velcro™ Strips around each footrest hangar bracket tube where the Airmesh Calf Strap is to be located.
3. Wrap the flexible part of the Airmesh Calf Strap over each Velcro™ Strip (this prevents sliding), with the padded side facing the body.

Cleaning: Machine wash warm (90° F, 32° C) with mild detergent. Do not use bleach. Line dry.

Warranty: All Motion Concepts products carry an unconditional guarantee against defects in workmanship and materials. Items found to be defective will be repaired or replaced at the discretion of Motion Concepts.

84 Citation Drive, Units 1-7, Concord, Ontario L4K 3C1
Toll Free Tel: 1.866.748.7943 Fax: 905.695.0

700 Ensminger Rd. Suite 112, Tonowanda, New York 14150
Toll Free Tel: 1.888.433.6818 Toll Free Fax: 1.888.433.6834

