


INSTALLATION INSTRUCTIONS for the MaTRx-V/Kid-a-bra Base

Our drop base cushions are designed for installation directly on to the wheelchair seat frame. Please read through the following instructions prior to installation:

1. Three styles of hooks are offered for use with our drop base cushions:
 - **Flushmount Hooks (0mm); 1" (25mm) Drop Hooks; and 2" (51 mm) Drop Hooks** (see **Figure 1.0** below);
 - The drop hooks are also available in **two sizes; 7/8" (22mm) and 1" (25mm)** to correspond with the tube diameter of the wheelchair frame.
2. Remove any existing seat cushions and/or seat upholstery from the seat frame.

Note: Two optional mounting locations are provided for each drop hook in order to avoid interference with any pre-existing obstructions on the wheelchair frame.

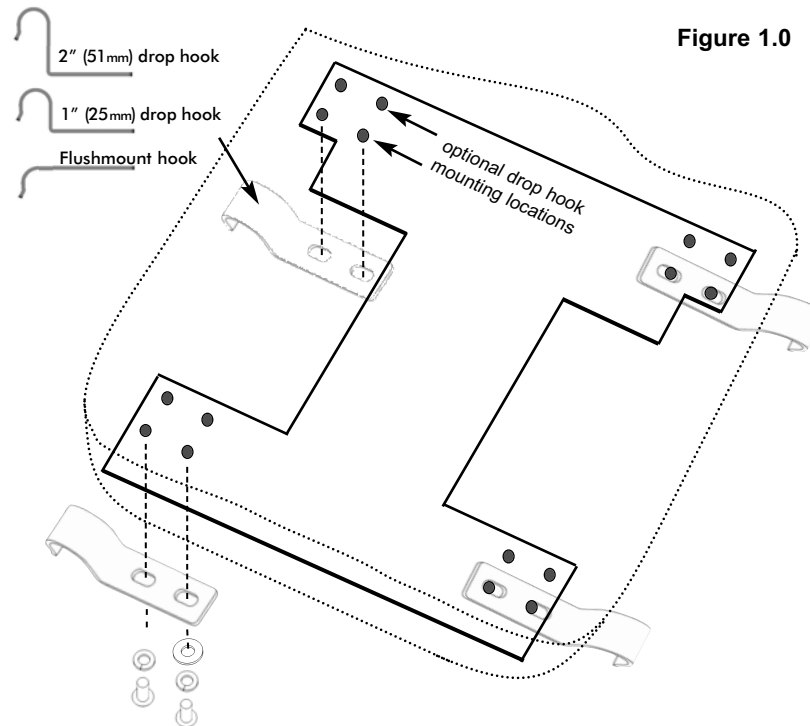

Note: In order to maintain the integrity of the inner cushion cover, the fabric is **not** pre-cut at the factory to reveal the location of the mounting holes- holes must be cut at the time of installation.

3. Estimate the final mounting position of the base on the wheelchair frame, and identify the appropriate mounting locations for all 4 drop hooks.

INSTALLATION INSTRUCTIONS for the MaTRx-V/Kid-a-bra Base

4. Locate the desired mounting holes through the inner cover fabric, and use a punch (or similar tool) to poke mounting holes through the fabric for each drop hook.
5. Fasten* each drop hook into position on the bottom of the base using the hardware provided (refer to **Figure 1.0**).
***Do not fully tighten hooks at this time.**
6. Place the drop base cushion onto the wheelchair and snap the drop hooks into place on the seat rails/tubes. (**note- flushmount hooks will rest on top of seat rails**).
7. Adjust the cushion (base) and drop hooks to the desired position on the wheelchair seat rails, then fully tighten all hooks.
8. A set of 'DuraClips' (2 pcs) is provided to help secure the drop hooks onto the wheelchair seat rails/tubes. The DuraClips install over the front drop hooks and contain a lock & release mechanism to allow the cushion to be removed when necessary. Cushions assembled with flushmount hooks also include a set of 'FixClips' (2 pcs) to secure the rear drop hooks onto the seat rails.

***** Detailed installation instructions are provided ***
for reference with each set of locking clips**

i Important! To ensure the safety of the client and prevent the cushion from lifting, please install all locking 'clips' provided.



U.S.: 700 Ensminger Rd., Suite 112, Tonawanda, NY 14150
TOLL FREE TEL: 1.888.433.6818 TOLL FREE FAX: 1.888.433.6834

Canada: 84 Citation Dr., Concord, ON L4K 3C1 Canada
TEL: 905.695.0134 FAX: 905.695.0138

