User Manual

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

Yes, you can.

Invacare[®] Matrix[®] Elite

Model Numbers PBE, PBE-HD, EDB,
EDB-HD, PBE-TR and PBE-TRHD

matrix[®]
SEATING SERIES

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

DANGER

Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Overview

Important Information

The best way to avoid problems related to pressure sores is to understand their causes and your role in a skin management program.

Your therapist and physician should be consulted if you have questions regarding individual limitations and needs.

All cushions should be selected carefully. Working with your therapist and physician is the best way to assure that a cushion choice matches your individual needs.

As the needs of the individual become more complex, cushion evaluation becomes more important.

Invacare Matrix Elite Backs

Our line of Invacare Matrix Elite backs are contoured to allow precise orientation within the wheelchair and provide optimal postural support. The foam is oversized to provide extra comfort and protection.

The Invacare Matrix Elite back includes a movable foam lumbar pad that can be installed/inserted behind the existing foam cushion to provide additional positioning capability. Refer to [Lumbar Support \(Optional\)](#) on page 12 for instructions on how to insert and adjust your lumbar support.

Outer Cover

The outer cover is made of a mesh material that is moisture resistant and breathable. Regular cleaning and inspection of the outer cover is recommended. Refer to [Care and Maintenance](#) on page 14.

2 Safety

-  Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Transporting the Client Within a Wheelchair Fitted with a Invacare Matrix Elite Back in a Motor Vehicle

The following guidelines should be made available to all parties who are responsible for the transportation of the client, such as schools and transport providers.

Whenever practicable the client should be transferred out of the wheelchair into a vehicle seat and use a lap and diagonal seat belt or an appropriate child safety seat.

A standard Invacare Matrix Elite deep back (EDBI820) fitted with a Motion Concepts PHRK headrest and standard Easy-Set attachment hardware has been dynamically tested and meets the requirements of ISO 16840-4 "Wheelchair Seating- Part 4: Seating Systems for Use in Motor Vehicles" and Section 20, RESNA WC/Volume 4 "Wheelchair Seating Systems for Use in Motor Vehicles". A forward facing frontal impact test was conducted using an ISO/RESNA Surrogate Wheelchair Frame (SWCF) with a surrogate metal seat, a 3" (76mm) thick cushion and a Hybrid III Midsized-Adult Male Anthropomorphic Test Device (ATD); ADT Weight = 170 lb (76.4kg). The SWCF was secured by a four-point, strap type wheelchair tie-down and the occupant restraint was a surrogate three-point belt with a SWCF anchored lap belt.

Invacare Matrix Elite backs should only be used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176-19.

Before Entering the Vehicle Check the Following:

Matrix Back - must be securely fastened into the wheelchair as described in the user manual.

Posture Belts and Harnesses - should remain fitted during transportation, although they are designed to provide postural support only, they are not intended for safety during transportation.

Head Support - must be securely fitted and correctly adjusted close to the rear of the client's head during transportation, in order to reduce the risk of whiplash injuries.

The Wheelchair Tie-down and Occupant Restraint System (WTORS)

Wheelchair Tie-down - The wheelchair must be securely tied down within the vehicle, in a forward facing position, in accordance with the wheelchair and tiedown manufacturers instructions.

Occupant Restraint - A suitable occupant restraint system must also be fitted in accordance with the manufacturer's instructions. A minimum is a lap and diagonal belt (lap belts on their own are not suitable). Restraints that loop over the shoulder and anchor to the vehicle floor should be avoided where possible, as they can cause a heavy downward load through the client during an impact, the preferred type anchor above and behind the shoulder as with a car seat belt. The lap section of the belt should fit snug over the pelvis of the client and should not be able to ride up to the abdominal area. The upper torso section of belt should be in contact with the client's chest and fit over the shoulder(s), while not cutting into the neck or slipping off the shoulder(s).

-  If the wheelchair and Matrix back are to be transported unoccupied, the wheelchair should be tied down with suitable equipment according to the manufacturer's instructions. If the equipment is to be dismantled for transportation, ensure that all parts are secured safely within the vehicle.

2.2 Intended Use

Our Invacare Matrix Elite backs are designed to provide precise orientation within the wheelchair for optimal postural support.

2.3 Wear and Tear Items

Normal wear and tear items and components include but are not limited to: all upholstery items including back upholstery and cushions.

Invacare reserves the right to ask for any item back that has an alleged defect in workmanship. See Warranty policy shipped with the product for specific warranty information.

Refer to User Manual for proper preventative maintenance schedule and use of the product.

This is just a general guideline and does not include items damaged due to abuse and misuse. Useful life is dependant upon many factors.

PRODUCT TYPE	PRODUCT WEAR AND TEAR PART
UPHOLSTERY AND SEATING:	Back Cushion Foam • Back Cushion Covers • Lateral Support Foam and Cover

2.4 General Warnings

DANGER

Risk of Death, Serious Injury or Damage

Improper use of this product may cause injury or damage.

If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment.

DANGER

Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury or damage.

Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using this wheelchair.

ACCESSORIES WARNING

Risk of Serious Injury or Damage

Use of non-Invacare accessories may result in serious injury or damage.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

DO NOT use non-Invacare accessories.

To obtain Invacare accessories, contact Invacare by phone or at www.invacare.com.

WARNING

Skin condition should be checked very frequently after the installation of any new seating system. Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.

Working with your therapist, physician, and equipment supplier is the best way to assure that a seating choice matches your individual needs.

As the needs of the individual become more complex, the seating evaluation becomes more important.

INSTALLATION WARNING

The procedures in this manual should be performed by a qualified technician.

Risk of Serious Injury or Damage

Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.

After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

DO NOT install the Invacare Matrix Elite back assembly onto back canes with an outside diameter greater than 1-inch or less than 3/4-inch. Otherwise, injury or damage may occur.

The mounting position of the Invacare Matrix Elite back is directly related to the chair's stability. When the Invacare Matrix Elite back is added to a TILT and/or RECLINING chair, it may cause a decrease in the chair's stability. It may be necessary to reposition the FRONT CASTERS, REAR WHEELS, BACK ANGLE, TILT-IN-SPACE, RECLINE POSITION and/or SEAT DEPTH before use. Use extreme caution when using a new seating position.

Heavy-Duty Mounting Hardware (HDMH) must always be used in combination with all models of Invacare Matrix Elite Heavy-Duty backs.

When an Invacare Matrix Elite is added to a TILT and/or RECLINING chair, a RIGIDIZER BAR or CROSSBAR must be present to stabilize the back canes.

Ensure the Invacare Matrix Elite back is properly secured to the wheelchair before using. Otherwise injury or damage may occur.

2.5 Invacare Matrix Elite Back Weight Limitation and Model Numbers

Refer to the chart for the weight limitation and stock model numbers.

Configured model numbers are: PBE and EDB

PBE = Elite Back, PBE-HD = Elite Back Heavy Duty, PBE-TR = Elite TR Back, PBE-TRHD = Elite TR Back Heavy Duty, EDB = Elite Deep Back, EDB-HD = Elite Deep Back Heavy Duty

CONFIGURED MODEL		DESCRIPTION	CONTOUR DEPTH	WEIGHT CAPACITY
STD	PBEwwhh	Elite Back	3 in. (76mm)	300 lbs. (136 kg)
	EDBwwhh	Elite Deep Back	6 in. (152mm)	
	PBEwwhh-TR	Elite TR (Tilt/Recline) Back	3 in. (76mm)	
HD	PBEwwhh-HD	Heavy Duty Elite Back	3 in. (76mm)	500 lbs. (227 kg)
	EDBwwhh-HD	Heavy Duty Elite Deep Back	6 in. (152mm)	
	PBEwwhh-TRHD	Heavy Duty Elite TR (Tilt/Recline) Back	3 in. (76mm)	

* ww hh = width/height

Back sizes (width/height) will vary depending on the model of Elite Back. Please contact your dealer or Therapist to determine the appropriate back model/size for your individual needs.

For further information on any of our Matrix backs, including all available models/sizes, please visit www.motionconcepts.com or www.invacare.com.

3 Setup

3.1 Hardware Kit Contents

If any of the hardware is missing or misplaced, please contact our customer service department and arrangements will be made to send you the necessary replacements.


Motion Concept - Canada: 866-748-7943 USA: 888-433-6818

Invacare - Canada: 800-668-5324 USA: 800-333-6900

QTY	DESCRIPTION
2	Back Mounting Plates
2	Cane Clamps
2	Cane Inserts (3/4" (19 mm))
2	Cane Inserts (7/8" (22 mm))
6	Depth/Recline Adjustment Screws
6	Cane Clamp Screws
2	Set Screws
2	Cane Clamp Nut Plates
4	Spacers** (not shown)
4	Long Mounting Pins (optional)
1	Elite Headrest Adapter Plate**
4	Truss Screws**
4	Locknuts**

* Parts shipped with 12 inch (31 cm) Elite backs only. One spacer ships installed between the back mounting plates and cane clamps. Two more are provided with the back.

** Parts shipped with medium, large and heavy duty Elite backs only.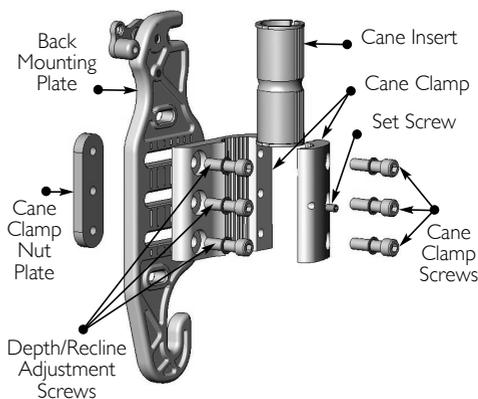

3.2 Tools Required

*Wrenches: 7/8" (22 mm)

*Allen Keys: 2mm, 4mm, 5mm

Utility Knife/Scissors

Tape Measure/Ruler

* Tools provided in kit


3.3 Before Installation

12 Inch (31 cm) Wide Elite Back Only - Spacer Installation

- i** Spacers are provided to fit the Elite back to the wheelchair back canes as follows:
 - 11 inch (28cm) to 12 inch (31cm) Back Cane Width - one spacer (shipped pre assembled between the cane clamp and back mounting bracket).
 - 12 inch (31cm) to 13 inch (33cm) Back Cane Width - Requires installation of a second spacer to each back mount.

1. Remove the three screws and cane clamp nut plate securing the back mounting plate from the cane clamp.
2. Position the dilatational spacer between the cane clamp and the back mounting plate.
3. Using the three screws and cane clamp nut plate, secure the cane clamp and two spacers to the back mounting plate.
4. Repeat STEPS 1-3 to install a second spacer between the remaining back mounting plate and cane clamp.

- i** When installing the Elite back on wheelchairs with a 12 inch (31cm) to 13 inch (33cm) back cane width long mounting pins must be installed. Refer to [Mounting Pin Configurations/Setup](#) on page 9.

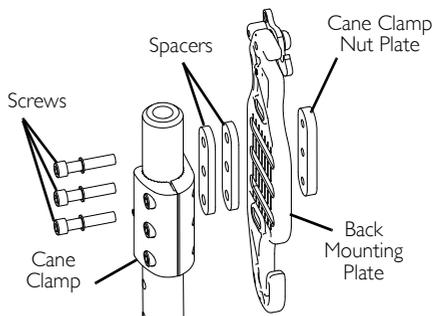Mounting Configuration

WARNING

When establishing the mounting position for the Invacare Matr^x Elite back, ensure the stability of the wheelchair is not compromised.

Adjusting the forward and aft position of the backrest will alter the user's center of gravity within the wheelchair. A recessed back position can significantly reduce the rearward stability of the wheelchair. Similarly, a more forward back position will reduce the wheelchair's forward stability.

The mounting hardware provided with your Elite back is pre-assembled in the rearmost mounting configuration. This allows the wheelchair seat depth to be maximized (up to a maximum 2.5" (64 mm) rearward offset - see image A).

By changing the orientation of the cane clamps on the mounting plates, the assembly can be reconfigured* to provide a more forward mounting position for the back if desired (up to a maximum 2.5" (64 mm) forward offset - see image B).


- i** Each mounting plates is also designed to provide 1.5" (38 mm) of fore and aft adjustment relative to the cane clamp.

*To reconfigure a mounting plate/cane clamp to the forward most orientation:

1. Detach the mounting plate from the cane clamp (3 screws);
2. Rotate the cane clamp 180° and re-fasten it to mounting plate;
3. Install cane clamp as illustrated in image B.

A


Front of Wheelchair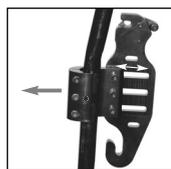

Mounting plate in rearmost position (maximized seat depth)

B

Front of Wheelchair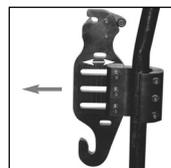


Mounting plate in forwardmost position

Accommodating Back Cane Size

i On some wheelchairs, there may be an uneven seam along the back posts where a smaller diameter tube is inserted into a larger diameter tube. If the cane clamps must be mounted over this seam, the plastic inserts may be cut in half to allow the clamp to fasten securely onto the back posts (see image B).

For wheelchairs with short back canes, it may be necessary to install Matrix Posture Back Cane Replacements (PBCR's). The PBCR's are designed to replace existing push handles or cane extensions on standard 1" (25 mm) back posts. The cane clamps can be mounted directly onto the replacement back canes in order to allow proper installation of the Invacare Matrix Elite back. Cane replacements are available from Motion Concepts (part#-TRX2810).

- 1" (25 mm) back cane - Cane clamps are installed without plastic inserts.
- 7/8" (22 mm) back cane - Requires 7/8" (22 mm) plastic inserts.
- 3/4" (19 mm) back cane - Requires 3/4" (19 mm) plastic inserts.

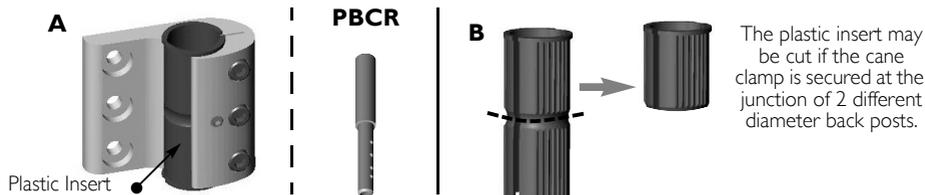3.4 Installing the Invacare Matrx Elite Back

Installing the Mounting Plates

WARNING

The installation hardware provided is high strength and tested for durability. **DO NOT** substitute hardware. Use only the hardware supplied.

1. Remove existing wheelchair back upholstery (if applicable).
2. Establish the desired mounting plate set-up/configuration for the user. Refer to [Mounting Configuration](#) on page 6.
3. Loosen cane clamp screws (x3) and loosely install the cane clamp (with mounting bracket) onto each back post. (Install plastic inserts if required, refer to [Accommodating Back Cane Size](#) on page 7).
4. Starting on one side, position the cane clamp at the desired height on the back post and tighten the clamp screws enough to hold it in place. Clamps and set screws will be fully tightened after the back is installed and all final adjustments have been made.
5. From a fixed point on the wheelchair (e.g. seat frame) measure the exact height of the installed cane clamp assembly, and install the second cane clamp at the same height on the opposite back post - see image A. Ensure the cane clamps/mounting plates are mounted parallel with each other.



Additional upper and lower adjustment slots are provided on the mounting plate to allow alternate mounting heights if obstacles are encountered. Using these alternate mounting slots will increase or decrease the back height by 7/8" (22 mm). These alternate slots will limit the range of available depth and/or recline angle adjustment.

6. To install the Elite back onto the mounting plates:
 - A. Slide the lower mounting pins into the hooked slots at the bottom of each mounting plate (see image B).
 - B. With the lower pins in place, lock/snap the upper mounting pins into the latch mechanism at the top of the mounting plates.


A Front view of wheelchair with back removed

Install cane clamps at equal heights and parallel to each other.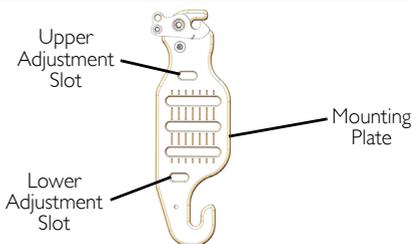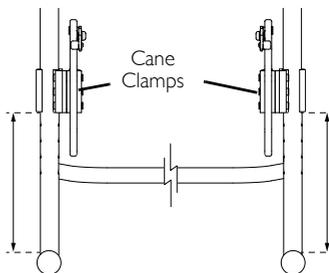

B

Upper Mounting Pin

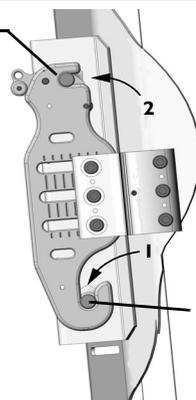Lower Mounting Pin

Mounting Pin Configurations/Setup

The Standard Elite back models are available in a range of widths up to 20W (51 cm). Heavy-Duty Elite back models range from 21W (53cm) - 24W (61 cm). All Elite back models are offered in 1" (25mm) width increments.

Two sets of mounting pins (standard pins and long pins) are supplied with every Elite back. The standard pins are factory installed and suitable for most wheelchair configurations. The long pins are designed to allow installation onto wheelchair back canes that are up to 1" (25 mm) wider than the width of the actual backrest:

Example- for a 16W (41 cm) Elite Back:

- 16" (41 cm) wheelchair seat frame/back cane width = Standard Pins
- 17" (43cm) wheelchair seat frame/back cane width = Long Pins

When a narrower mounting pin configuration is used, the range of depth/recline adjustment for the back may be more limited than in wider configurations.


Extended Mounting Pins (EMP) are also available (sold separately) to allow interfacing of Elite backs onto wheelchair seat frames up to 2" (51 cm) wider than the actual back width- see Warning below.

12 INCH (31 cm) WIDE ELITE BACK (see image A) - Ensure the correct number of spacers have been installed between the back mounting plate and cane clamp (refer to [12 Inch \(31 cm\) Wide Elite Back Only - Spacer Installation](#) on page 6).


When installing the Elite back on wheelchairs with a 12 inch (31cm) to 13 inch (33cm) back cane width long mounting pins must be installed.

ELITE HD BACK (see image B) - A pem-nut backing plate (not shown) is used to secure the mounting pins to the Elite HD back. When exchanging the mounting pins the pem-nut backing plate must be realigned with the new mounting pins to complete installation.


Rear View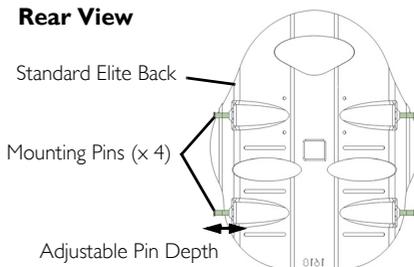

A

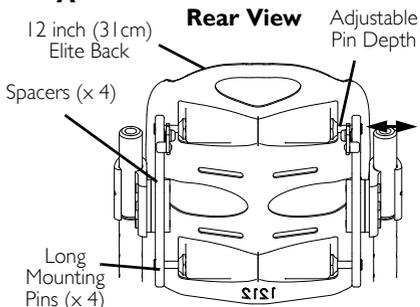B Rear View

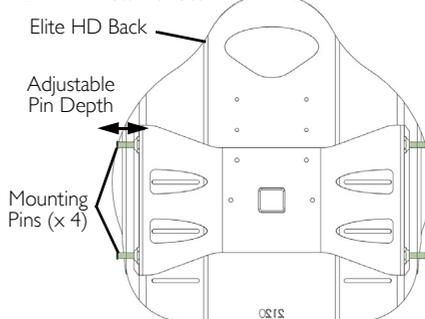WARNING

Extended Mounting Pins (EMP's) are only for use with our standard back models (up to 300 lbs/136 kg weight capacity).

Never use EMP's with Invacare Matrix Heavy-Duty Elite back models; EMP's are not recommended for use in tilt applications.

Mounting Pin Adjustment

In most instances, the depth of the mounting pins will require some minor adjustment to properly complete the installation of the Elite back onto the wheelchair back canes.

CAUTION



Improper pin set-up may cause damage to the back pan and/or the mounting plates. Ensure sufficient spacing is provided between the mounting plates and the mounting pins (a min. gap of 1/8" (3mm) is recommended).

Ensure flange nuts are fully tightened; Failure to do so may cause damage/cracking in the back pan.

Ensure pins do not rotate when tightening the flange nuts.

NEVER use Loctite® on mounting pin hardware.

1. With the back secured in place on the wheelchair, determine the amount of adjustment required for each mounting pin.
2. Remove the backrest from the wheelchair and use a wrench to loosen the flange nut on each pin that requires adjustment. If necessary, replace the standard mounting pins with long mounting pins.
3. Adjust the mounting pin depth (clockwise to shorten/counter-clock wise to lengthen) so that a minimum gap of 1/8" (3mm) exists between the top of the pin and the mounting plate.
4. Re-tighten each flange nut to secure the pins in place.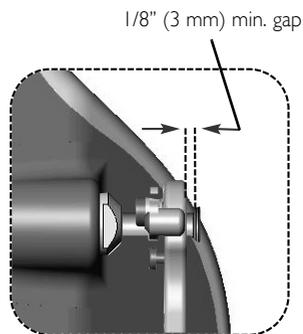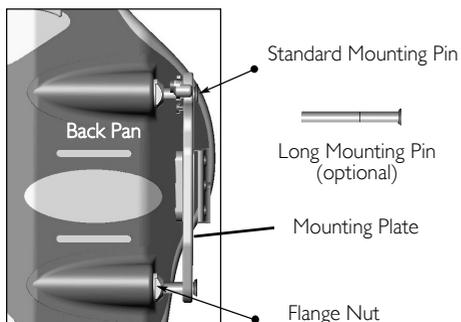


Installing the Elite Back

1. Verify that the overall fit and alignment of the Elite back is correct (make adjustments as needed).
2. Fully secure the cane clamp screws onto both back posts, and tighten the set screws to ensure the mounting plates do not slip or rotate.
3. To best determine the optimal depth and recline settings for the Elite back, the user should be seated in the wheelchair. While seated, loosen the depth/recline screws on the mounting plates and adjust the Elite back to the desired depth and recline angle. Re-tighten all screws to lock the back in position.

- i** Ensure that the depth/recline adjustment for the Elite back is made symmetrically* on the left and right mounting plates.

The vertical score lines on the mounting plates are designed as a point of reference to assist with symmetrical mounting. Since the Elite back can be mounted in a rotated (reclined) position if desired, it will not always be mounted parallel with these vertical lines.

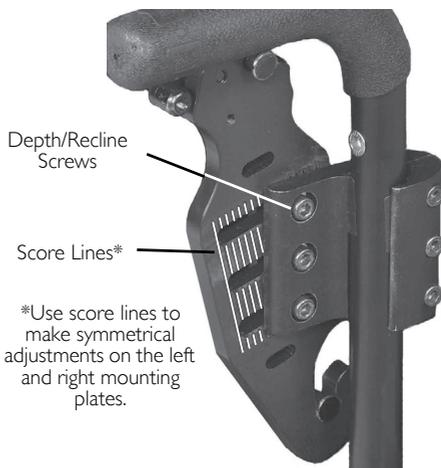3.5 One-Hand Cord Release Mechanism

1. Thread the nylon cord through the holes in the latch mechanism at the top of the left and right mounting plates (see image A).
2. Install the cord lock mechanism onto the cord (see image B). Press the button to open the lock mechanism and slide the two open ends of the nylon cord through the opening.
3. With the back properly installed on the wheelchair, adjust the cord length so that it hangs at approximately a 45° angle along the back pan (this allows the release mechanism to operate more effectively). If necessary, cut/remove excess cord to eliminate any risk of entanglement. Once adjusted, combine the loose cord ends into a single knot to prevent the cord-lock from slipping.

- i** If the cord length was modified, carefully melt the ends of the cord to prevent fraying.

Thread cord through the latch mechanism.


Tie a knot using both cord ends.

Press button to open/adjust.

Melt cord ends to prevent fraying.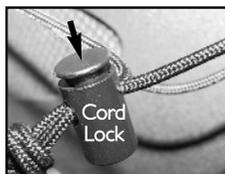



3.6 Installing the Invacare Matrix Elite Headrest Adapter Plate

i Headrest adapter plates are shipped with all Elite back models ranging from 13 inch (33cm) to 24 inch (61cm) wide.

Headrest adapter plates are available as an option for 12 inch (31cm) wide Elite backs.

The following instructions apply to installing a headrest adapter plate on any size Elite back.

1. Release the hook and loop fasteners and remove the outer back cover.
2. Align the headrest adapter plate with the four mounting holes in the center of the Elite back.
3. Using the four truss screws, two spacers (large Elite HD backs only) and four locknuts, secure the headrest adapter plate to the Elite back. Securely tighten.
4. Reinstall the outer cover.

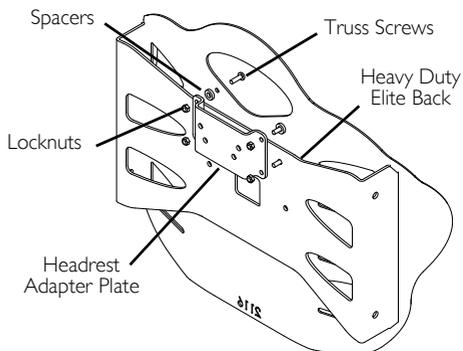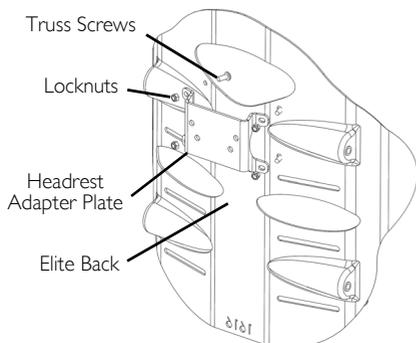3.7 Lumbar Support (Optional)

i The Lumbar Support is a pre-fabricated foam insert that provides lower back support for additional comfort. The position of the lumbar insert can be adjusted inside the Elite back cover, or it may be removed if no lumbar support is desired.

Lumbar Support Adjustment

1. To access the Lumbar Support, open the hook and loop fastening strips on the outer cover at the bottom rear of the back pan.
2. The foam lumbar support/insert is installed inside the Elite back cover between the foam back cushion and the back pan (see image B).
3. The lumbar support can be adjusted manually to any desired height/position.
4. Refasten the cover onto the back pan.

3.8 Lateral Supports (Optional)

Lateral Pads are available as an option for additional trunk support. Lateral Pads are offered in standard (small, large) and offset styles (see image A). Mounting slots in the back shell and the lateral support brackets allow for adjustment both vertically and laterally. Upper and lower mounting slots are provided on the back pan for an even greater range of adjustment. Lateral Supports may be further customized by carefully bending* the lateral to a more desirable contour/position for the client.

WARNING

*For best results and to prevent damage, the lateral supports must be removed from the back pan prior to bending

Installing Lateral Supports


1. Pull up the bottom of the cover (release hook and loop fasteners) to expose the mounting slots/channels in the back pan.
2. Determine the desired mounting height/orientation for the lateral pads (upper or lower mounting slot position).
3. If applicable, carefully bend the lateral support(s) to the desired contour (prior to installation).
4. Loosely install the lateral supports onto the back pan using the hardware provided (see image B).

Hardware can be installed in either orientation on the back pan (with the backing plates located on the inside or outside of the back pan). The standard mounting orientation is shown in image B.


5. Adjust laterals to the desired position via the mounting slots on the backpan and via the slots in the lateral support bracket. Secure each lateral support into place. (see image C)
6. Re-attach the back cover via the hook and loop fasteners.

A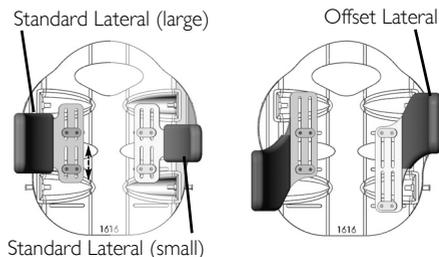


Offset laterals may be mounted to the back pan in either orientation (as shown) to provide extended support.

B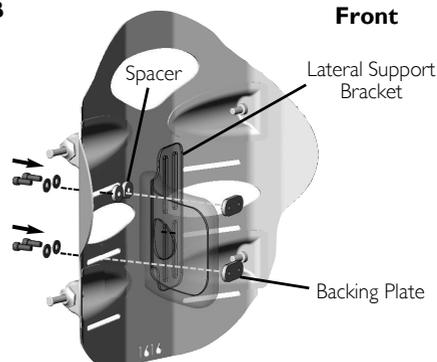

Spacers are only required when using the upper slot on the back pan.

C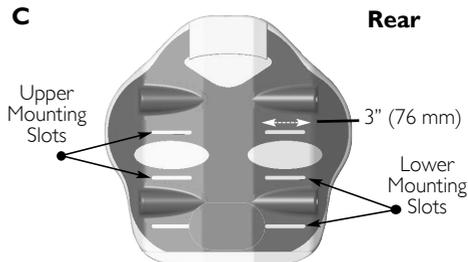


Lower mounting slots not available on 10" (26 cm) or 12" (31 cm) high Invacare Matr[®] Elite backs.

4 Usage

4.1 Disengaging the Back from the Wheelchair

1. Disengage the upper latch mechanisms and release upper mounting pins as follows:
 - No Cord Release Mechanism - Manually lift the release levers (left and right) to unlock the latch mechanisms and release the upper pins from the mounting plates as shown.
 - Cord Release Mechanism - Lift cord upward to simultaneously unlock both latch mechanisms. Refer to One-Hand Cord Release Mechanism on page 11.
2. Lift entire backrest up and forward to disengage the lower pins from the lower slots in the mounting plates.

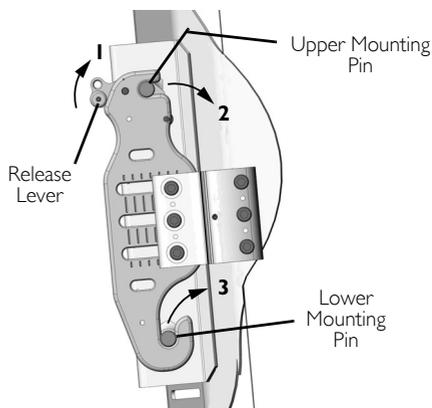5 Care and Maintenance

5.1 Inspection

WARNING

Check ALL fasteners weekly to ensure that mechanical connections and attaching hardware are tightened securely - otherwise injury or damage may occur.
DO NOT continue to use this product if problems or damage are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

Visually inspect parts including upholstery materials, foams (if accessible), and plastics, for deformation, corrosion, breakage, wear or compression.

5.2 Cleaning

WARNING

To avoid infectious exposure, inspect cushion thoroughly and properly clean and disinfect any materials soiled with blood or other bodily fluids.

CAUTION

DO NOT use cleaning agents containing phenols, alcohols bleaches or other abrasive materials. Otherwise product damage may occur.

1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly. Do not use granules.

Keep clear of open heat sources.

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

For further information on decontamination in health care environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.

Cleaning (Light Soilage) - Outer and Inner Cover

1. Wipe down the outer cover with a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the outer cover with clean water using a single use nonabrasive cloth.
3. Dry the outer cover thoroughly before use.

Cleaning (Heavy Soilage) - Outer Cover Only

Where the outer cover is badly soiled, we recommend cleaning with a dilute cleaning solution at 176 °F / 80 °C in the washing machine.

Remove the foam cushion before washing the outer cover. **DO NOT** place the foam cushion into the washing machine.

Large spillages of bodily fluids should be absorbed and removed with paper towels first, before removing and washing the cover.

1. Clean up all spillages of bodily fluids i.e. blood, urine, faeces, sputum, wound exudate and all other bodily secretions as soon as possible using a 1% Chlorine Solution (10,000 ppm).
2. Rinse the outer cover with clean water using a single use nonabrasive cloth.

Polyurethane coated fabrics can absorb liquids for short periods causing a temporary change to the polyurethane characteristics. The cover will swell temporarily and is more vulnerable to physical damage for a period after it is completely surface dried, by which time it will revert to its previous state.

Tumble dry setting must not exceed 122 °F /50 °C.

Do not tumble dry for longer than 10 minutes.

Thoroughly dry cover before re-fitting to the foam cushion.

6 After Use

6.1 Reuse

WARNING: Risk of Injury

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, **DO NOT** use the product. Otherwise injury may occur.

A cleaning record must be kept as part of the cleaning system.

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

It is recommended that the Elite back is checked (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the release of each patient by a suitably qualified and competent person.

Before reuse, clean the product thoroughly, Refer to Cleaning on page 14.

6.2 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

7 Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare/Motion Concepts warrants this product to be free from defects in materials and workmanship for two years of use by original purchaser. This warranty does not apply to punctures, tears or burns, nor to the removable cushion cover. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's/Motion Concepts' option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's/Motion Concepts' sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare/Motion Concepts product. In the event you do not receive satisfactory warranty service, please write directly to Invacare/Motion Concepts at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation/Motion Concepts will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S/MOTION CONCEPTS' EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE/MOTION CONCEPTS; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S/MOTION CONCEPTS' CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE/MOTION CONCEPTS DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE/MOTION CONCEPTS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE/MOTION CONCEPTS SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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Canada

84 Citation Dr., Units 1-9
Concord, Ontario L4K 3C1
905-695-0134

© 2015 Motion Concepts. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Motion Concepts. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Motion Concepts unless otherwise noted. Loctite is a registered trademark of Henkel Corporation.

Part No 1171724

Rev B-06 - 10/27/15

Making Life's Experiences Possible™