# Extended Mounting Pin (EMP) Kit Instructions

EN

The components utilized in this Extended Mounting Pin Kit will vary depending on the type of Matrix Back cushion they are being applied to. For instructions on installing the EMP's onto the Matrix-PB Elite back, please refer to section I. below. For instructions on installing the EMP's onto the Matrix-PB back, please refer to section II.


**⚠ WARNING!** EMP's are designed only for use with our standard back sizes (up to a max. 300lb. (136kg) weight capacity).

## I. EMP INSTALLATION INSTRUCTIONS for MATRIX-PB ELITE BACKS

1. Loosen the existing flange nuts and mounting pins (x4) from the Matrix PB Elite back shell.
2. Thread the new flange nuts (*provided*) onto the new *extended* mounting pins (x4) and loosely install them onto the Matrix PB-Elite back shell as indicated below.
3. Manually adjust the depth of the extended mounting pins to the desired depth (please see **Important!** note below).
4. With the pin depth properly adjusted, **fully secure** the flange nuts against the back shell using the wrench provided.

**i Important!** During final set-up of the MaTRx PB back, ensure a minimum gap of 1/8" (3mm) is maintained b/w the mounting plates and the head of the mounting pins

**i Important!** Ensure all flange nuts are fully tightened against the back shell, failure to do so may result in damage to the back shell and/or the mounting hardware.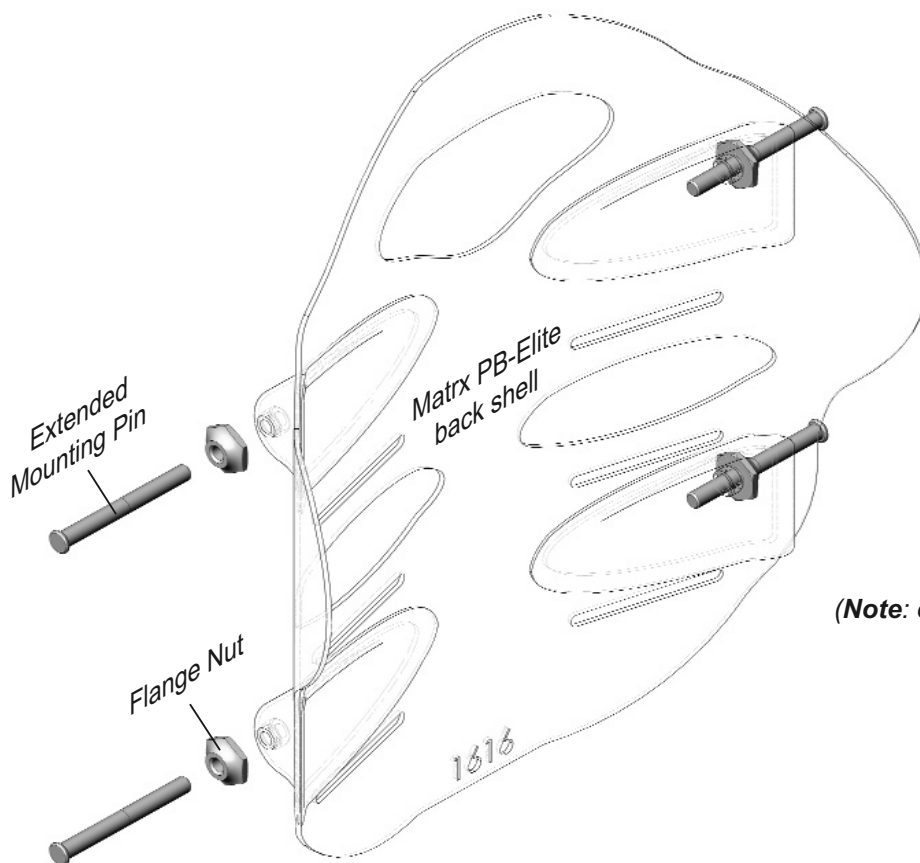


# Extended Mounting Pin (EMP) Kit Instructions

**ⓘ WARNING!** EMP's are designed only for use with our standard back sizes (up to max. 300lb. (136kg) weight capacity).

## II. EMP INSTALLATION INSTRUCTIONS for MATRX-PB BACKS

1. Remove the outer cover and moulded foam cushion from the MaTRx-PB back shell.
2. Remove the original mounting pins (x4) and their corresponding flange nuts from the MaTRx-PB back shell.
3. Working on one side of the back, place/align the reinforcement plate (*provided*) on the outside of the back shell as indicated in **Figure 1.0**, and manually install the new EMP's and flange nuts to hold the reinforcement plate in position.
4. Using the innermost holes on the reinforcement plate as a template (see **Figure 2.0** below), drill 2 pilot holes through the back shell and metal backing plate ( $1/8''$  (3mm) drill bit req'd).
5. Secure the reinforcement plate onto the back shell using the hardware provided. (refer to **Figure 3.0** below)
6. Repeat steps 3-5 to install the reinforcement plate onto the opposite side of the back shell.
7. With both reinforcement plates installed, manually adjust the depth of the EMP's to the desired depth (see **Important!** note below).
8. With the pin depth properly adjusted, **fully secure** the flange nuts against the back shell using the wrench provided.
9. Carefully reinstall the MaTRx-PB cushion foam and back cover onto the backshell.

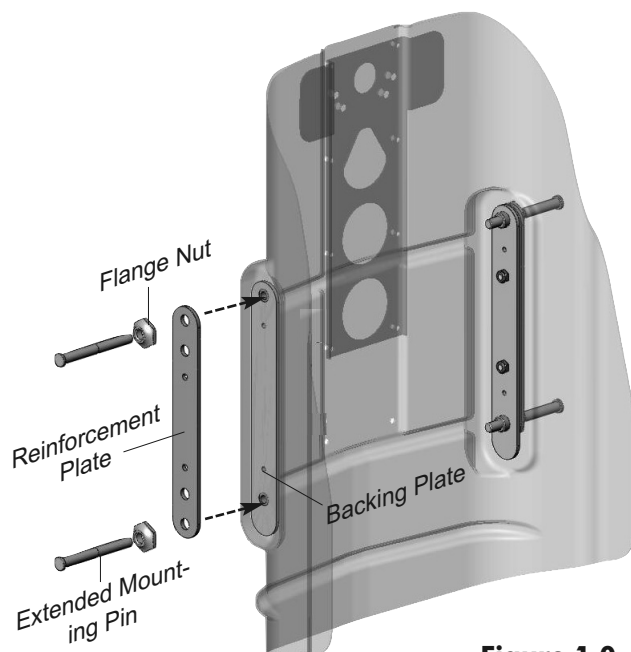
**Figure 1.0**

**ⓘ Important!** During final set-up of the MaTRx PB back, ensure a minimum gap of  $1/8''$  (3mm) is maintained b/w the mounting plates and the head of the mounting pins


**ⓘ Important!** Ensure all flange nuts are fully tightened against the back shell, failure to do so may result in damage to the back shell and/or the mounting hardware.

**Figure 3.0**

**⚠ WARNING!** Use caution when drilling holes into the back pan. Ensure the back is properly stabilized and your hands/ fingers are out of harms way prior to drilling.

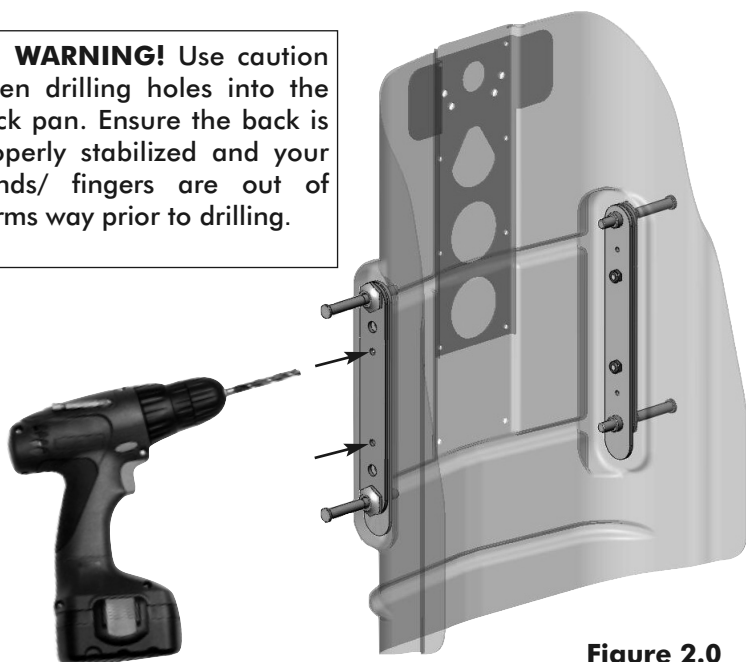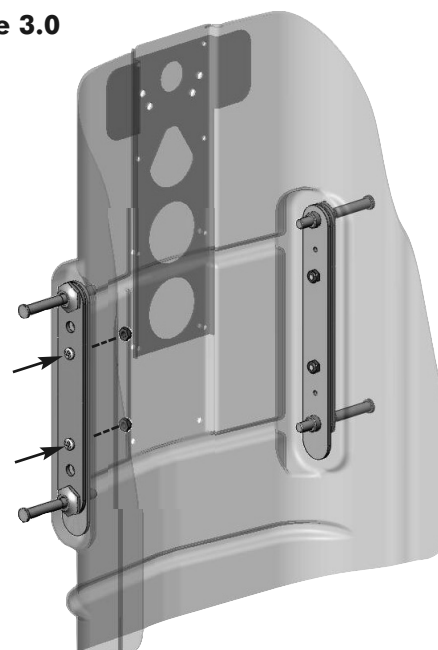**Figure 2.0**

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**Canada** • Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 •
**Danmark** • Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0) 36 90 00 00 •
**Sverige & Suomi** • Invacare AB, Box 66, S-163 91, Spånga SWEDEN • Tel: (46) (0) 8 761 7090 •
**Österreich** • Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee, Austria •