

5.0 - Mechanical Adjustments:

VIII. Single Post Back- Angle Adjustments

The TRx Single Post Back eliminates the need for back canes and is also capable of folding down to less than 11 inches off the seat pan. The back angle is adjusted manually with a total available **adjustment range of 36°**, from the most posterior position (18° recline) to the most anterior position (18° pre-cline). Adjustments are set at 6° increments. The back position is factory pre-set to the maximum (18°) reclined position .

To Adjust the Back Angle:

1. Lift the release handle to unlock the hinge mechanism.
2. Manually adjust the back to the desired angle/ position. There are seven hole positions available for adjustment. The back angles associated with each hole position are illustrated below. *(please note that the angles shown are approximate and are measured relative to the seat frame)* .
3. When the desired angle is established, return the handle to the locked position.

