Invacare® Matrix® E2S/E2D Back

en	Backrest User Manual	5
de	Rückenlehne Gebrauchsanweisung	11
fr	Dossier Manuel d'utilisation	17
it	Schienale Manuale d'uso	23
es	Respaldo Manual del usuario	29
pt	Encosto Manual de utilização	35
nl	Rugleuning Gebruiksaanwijzing	41
da	Ryglæn Brugsanvisning	47
no	Rygglene Bruksanvisning	53
sv	Ryggstöd Bruksanvisning	59
fi	Selkänöja Käyttööpas	65

en This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.

This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

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DEALER: This manual **MUST** be given to the user of the product.

USER: Before using this product, read this manual and save for future reference.

1.0 GENERAL WARNINGS/INFORMATION

1.1 OVERVIEW

SAFETY SYMBOLS:

Proper set-up and safe use of your Invacare Matrix E2S/E2D Back depends on your own good judgement and common sense, as well as that of your provider, caregiver and/or health professional. The following symbols are used throughout this user manual to identify warnings, cautions and important notes. It is very important that you read and understand them completely.

CAUTION! Failure to heed the cautions in this manual may result in damage to your Matrix Seating product.

WARNING! Failure to heed the warnings in this manual may result in death, serious injury or damage.

IMPORTANT! Important information to remember during installation, set-up and/or adjustment of you Matrix Seating product.

USER WARNING!

Improper use of this product may cause injury or damage. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.

IMPORTANT INFORMATION

The best way to avoid problems related to pressure injuries is to understand their causes and your role in a skin management program. Your therapist and physician should be consulted if you have questions regarding individual limitations and needs. All back supports should be selected carefully. Working with your therapist and physician is the best way to assure that a cushion choice matches your individual needs. As the needs of the individual become more complex, the evaluation becomes more important.

IMPORTANT! Please refer to the Wheelchair Owners/Operators Manual for additional safety information, warnings and cautions related to the operation of your wheelchair.

1.2 INTENDED USE

Our Invacare® Matrix® E2S and E2D back is intended to provide back support for a wide variety of wheelchair users. It is designed to allow precise orientation within a wheelchair to ensure optimal postural support and positioning. The E2S/E2D Back support can be installed on a wide range of wheelchairs.

2.0 SAFETY INFORMATION

2.1 TRANSPORTATION WARNINGS!

WARNING! Risk of Death, Serious Injury or Damage

- Risk of severe injury when transporting the client within a wheelchair fitted with an Invacare® Matrx® E2S/E2D Back in a motor vehicle.

TRANSPORTING THE CLIENT IN A MOTOR VEHICLE WITHIN A WHEELCHAIR FITTED WITH A MATRX BACK

The following guidelines should be made available to all parties responsible for the transportation of the client, such as schools and transport providers.

- Wherever practicable Motion Concepts recommends that the client is transferred out of the wheelchair into a vehicle seat using a lap and diagonal seat belt or an appropriate child safety seat.

- The following back supports have been dynamically tested and meet the requirements of ISO 16840-4:2009, Wheelchair Seating- Part 4: Seating Systems for Use in Motor Vehicles; and Section 20, ANSI/RESNA WC-4:2017, Wheelchair Seating Systems for Use in Motor Vehicles:

- Invacare® Matrx® E2S back with Removable Hardware and Elan Headrest
- Invacare® Matrx® E2S back with Fixed Hardware and Elan Headrest

A forward facing frontal impact test was performed for each configuration using an ISO/RESNA Surrogate Wheelchair Frame (SWCF) and a Hybrid III Midsize-Adult Male Anthropomorphic Test Device (ATD). The SWCF was secured by a four-point, strap type wheelchair tie-down and the ATD was restrained by a surrogate three-point belt with a SWCF-Anchored Lap Belt.

- Invacare® Matrx® E2S/E2D backs used to transport a client should be equipped with a Headrest Support (i.e. restricted to back heights 14in (36cm) and higher); and may **only** be used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176-19.

- For safe transportation of a client within a wheelchair in a motor vehicle, refer to the user manuals of the wheelchair and tie-down and occupant restraint system (WTORS)- see additional information below.

BEFORE ENTERING THE VEHICLE CHECK THE FOLLOWING:

The Matrx Back must be securely fastened into the wheelchair as described in the user manual.

Posture Belts & Harnesses should remain fitted during transportation, although they are designed to provide postural support only, they are NOT intended for safety during transportation.

A Head Support should be securely fitted and correctly adjusted close to the rear of the client's head during transportation, to reduce the risk of whiplash injuries.

WHEELCHAIR TIE-DOWN & OCCUPANT RESTRAINT SYSTEM (WTORS)

- **Wheelchair Tie-down:** The wheelchair must be securely tied down within the vehicle, in a forward facing position, in accordance with the wheelchair and tie-down manufacturers instructions.

- **Occupant Restraint:** A suitable occupant restraint system must also be fitted in accordance with the manufacturer's instructions. A minimum is a lap and diagonal

belt (lap belts on their own are not suitable). Restraints that loop over the shoulder and anchor to the vehicle floor should be avoided where possible, as they can cause a heavy downward load through the client during an impact, the preferred type anchor above and behind the shoulder as with a car seat belt. The lap section of the belt should fit snugly over the pelvis of the client and should not be able to ride up to the abdominal area. The upper torso section of belt should be in contact with the client's chest and fit over the shoulder(s), while not cutting into the neck or slipping off the shoulder(s).

i IMPORTANT! If the wheelchair and Invacare® Matrx® E2S/E2D back is transported unoccupied, the wheelchair should be tied down with suitable equipment according to the manufacturer's instructions. If the equipment is dismantled for transportation, ensure that all parts are secured safely within the vehicle.

2.2 GENERAL SAFETY AND INSTALLATION WARNINGS!

⚠ WARNING! Risk of Death, Serious Injury, or Damage

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury or damage. Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using this wheelchair

⚠ WARNING! Risk of Serious Injury

When establishing the mounting position for the Invacare® Matrx® back, ensure the stability of the wheelchair is not compromised. Adjusting the forward and aft position and/or angle of the backrest will alter the user's center of gravity within the wheelchair. It may be necessary to reposition the front casters, rear wheels, back angle, tilt-in-space, recline position and/or seat depth before use.

⚠ WARNING! Risk of Serious Injury or Damage

- The procedures in this manual should be performed by a qualified technician
- The hardware provided is high strength and tested for durability. **Do Not substitute hardware.** Use only the original hardware supplied with your Matrx back
- Attaching hardware that is loosely secured could cause loss of stability resulting in serious injury or damage.
- After ANY adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.
- DO NOT install the Invacare® Matrx® back assembly onto back canes with an outside diameter greater than 1" (25mm) or less than 3/4" (19mm). Otherwise, injury or damage may occur.
- For power seating applications, the Fixed Mounting Hardware must be used.

⚠ WARNING! Risk of Injury

Invacare® Matrx® products are specifically designed and manufactured for use in conjunction with Invacare® Matrx® accessories. Accessories designed by other manufacturers have not been tested by Motion Concepts and are not recommended.

⚠ WARNING Risk of Injury

Skin condition should be checked very frequently after the installation of any new seating system. Working with you therapist, physician and service provider is the best way to assure that a seating choice meets your individual needs.

3.0 MATRX E2S/E2D BACK INSTALLATION

3.1 MATRX E2 BACK INSTALLATION & SET-UP

❗ IMPORTANT! The installation and set-up of the Matrix E2S/E2D Back **MUST** be performed by a Qualified Technician and/or Healthcare Provider.

The Matrix E2S/E2D Back has been designed to install onto back canes that are up to 1" (25mm) wider or 1" (25mm) narrower than the nominal back width.

The type of mounting hardware required will vary depending on several factors, including the needs of the end user, the type of wheelchair being used, and the height of the E2 back shell being installed.

4.0 CARE AND MAINTENANCE

4.1 INSPECTION AND MAINTENANCE

Frequent visual inspection should be performed on all components, including upholstery, foam, hardware/fasteners and any plastic components.

⚠ WARNING! Risk of injury or damage

- Do not continue to use this product if problems or damage are discovered;
- Corrective maintenance can be performed by or arranged thru your local service provider

Weekly: Inspect **ALL** fasteners weekly to ensure that mechanical connections and attaching hardware are tightened securely.

Monthly: Perform a visual inspection of parts including hardware, brackets, upholstery materials, foam, and plastics for deformation, corrosion, breakage, wear or compression. Replace damaged/worn components if necessary.

4.2 CLEANING/LAUNDERING OF COVERS:

⚠ WARNING! Risk of damage

To avoid infectious exposure, inspect cushion thoroughly and properly clean and disinfect any materials soiled with blood or other bodily fluids.

⚠ CAUTION! Risk of damage

- DO NOT use cleaning and disinfecting agents with abrasive, staining, or polymer-damaging properties such as phenols, alcohols, or bleaches. Chlorine solutions used even in low concentrations on a regular basis can diminish the life of the cover.
- Chosen cleaning agents and disinfectants must be compatible with one another in order to effectively protect the materials on which they are applied.
- Keep clear of open heat sources during cleaning.

4.2.1 GENERAL CLEANING:

OUTER COVER: The Meshtex fabric used on the E2 Back cover can be wiped with a damp absorbent towel, cloth, or paper towel. Remove all gross contamination from surface (i.e. food particles, urine, feces, blood). Spot-clean with mild soap and water.

BACK SHELL/HARDWARE: The back shell and hardware/components can be wiped clean with soap and water as necessary.

4.2.2 LAUNDERING INSTRUCTIONS

- i IMPORTANT!** PRIOR to Laundering: The MAC Back cover **MUST** be removed from the inner foam cushion. (**Note:** Do Not Dry Clean, Do Not Bleach).
- i IMPORTANT!** Adequate drying is a crucial factor in the care and maintenance of all covers. The covers should be dried completely prior to use on the cushion. Air drying of covers is recommended.

I. MACHINE LAUNDERING or HAND-WASHING INSTRUCTIONS (OUTER COVER)

- Carefully remove the E2 Back cover from the back shell by releasing the velcro hook & loop fasteners along the top, bottom and inside of the back shell.
- Carefully remove the inner foam cushion from the inside of the E2 Back cover.
- Meshtex fabric covers can be hand-washed or machine-washed in warm water not higher than 60°C (140°F), on a delicate cycle with neutral detergents such as Tide or Persil.
- To avoid possible damage, ensure the cover does not come in contact with any other sharp garment accessories during laundering.

II. DRYING INSTRUCTIONS (OUTER COVER)

Adequate drying is a crucial factor in the care and maintenance of the outer cover. E2 covers should be dried completely prior to use. Air drying is the method of choice.

- DO NOT WRING OUT.
- Hang cover and allow it to drip dry to minimize wrinkling.
- Alternatively, covers may be tumble-dried on low/cool setting.
- DO NOT IRON OR PRESS Meshtex covers

- i** *Polyurethane (PU) coated fabrics can absorb liquids for short periods causing a temporary change to the PU characteristics. The wet cover is more vulnerable to physical damage and should not be used until fully dried.*

4.3 REUSE/RECYCLING

⚠ WARNING! Risk of Injury

Always have an authorized dealer inspect the product for damage or wear PRIOR to transferring the product to a different user. If any damage is found, DO NOT use this product. Failure to heed these warnings may result in severe injury.

This product is suitable for re-use. The outer cover (fabric and foam), back shell and mounting hardware must be inspected by a qualified dealer or healthcare professional before re-use. If any damage is found, DO NOT RE-USE

PRIOR to reuse, clean and disinfect the product thoroughly in accordance with local government regulations.

4.4 DISPOSAL

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations. Ensure that the back is cleaned prior to disposal to avoid any risk of contamination.

5.0 WARRANTY INFORMATION

WARRANTY

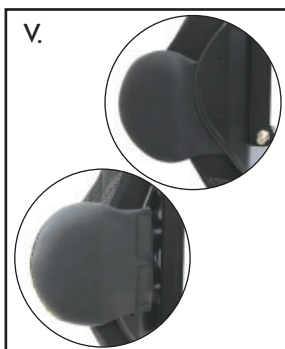
We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

6.0 MATRX E2 ACCESSORIES

6.1 ADDITIONAL E2 ACCESSORY OPTIONS (Sold Separately)

- I. PRIVACY FLAP (PFLAP)
- II. BACKBONE (PSIS/SacroLumbar Support)
- III. POSTURE FLEX (Abdominal Support) and QUAD RELEASE (QR) TAB
- IV. CHEST HARNESS INTERFACE
- V. EASYFIX LATERALS
- VI. SWING-AWAY LATERALS

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Invacare Limited
Invacare International GmbH
Advena Ltd.