Invacare® Matrix® Mini Back

en	Backrest	
	User Manual	2
de	Rückenlehne	
	Gebrauchsanweisung	7
fr	Dossier	
	Manuel d'utilisation	11
it	Schienale	
	Manuale d'uso	15
es	Respaldo	
	Manual del usuario	19
pt	Encosto	
	Manual de utilização	23
nl	Rugleuning	
	Gebruiksaanwijzing	27
da	Ryglæn	
	Brugsanvisning	31
no	Rygglene	
	Bruksanvisning	34
sv	Ryggstöd	
	Bruksanvisning	38

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

matrx®
SEATING SERIES

Yes, you can.®

en This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.

This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

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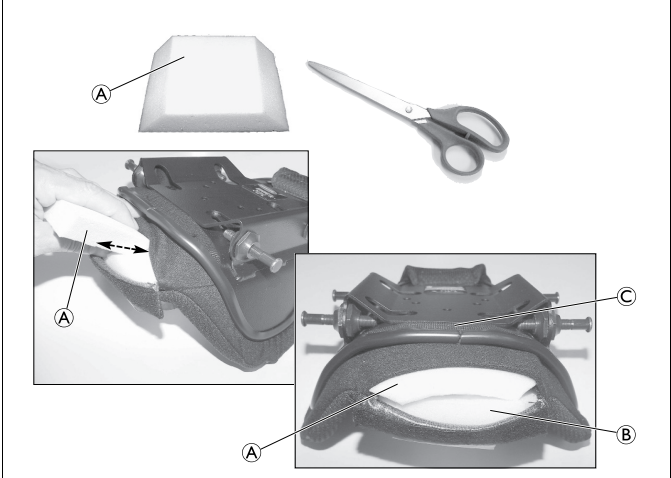
Fig. 11

1.2 Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

IMPORTANT

Indicates a hazardous situation which if not avoided could result in damage to the product.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

1.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

1.4 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

The product meets the requirements of the standards DIN EN 12182 (Technical aids for disabled persons - general requirements and test methods), DIN EN ISO 14971 (Medical devices - Application of risk management to medical devices) and DIN EN ISO 10993-1 (Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management system).

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.5 Intended use

The Matrx® Mini back is designed to be used on a pediatric wheelchair to maximize seating comfort.

1.6 Scope of delivery (Fig. 1)

The following components are included in the scope of delivery:

Pos.	Designation
A	Mini mounting plates (2x)
B	Cane clamp assembly (2x)
C	Cane plastic inserts Ø 22 mm (2x)
D	Cane plastic inserts Ø 19 mm (2x)
E	Depth/Recline adjustment screws (4x)
F	Cane clamp screws (2)
G	Cane clamp nut plates (2)

Pos.	Designation
	Long mounting pins (optional)
Tools	Wrench (22 mm) / Allen key (4 mm / 5 mm)

1.7 Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety information

WARNING!

– Do not use this product or any available optional equipment without first completely reading and understanding the user manual supplied. Invacare® product manuals are available on your local Invacare website or at your local dealer. If you are unable to understand the warnings, cautions or instructions please contact a health care professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

WARNING!

Risk of injury

– Invacare® products are specifically designed and manufactured for use in conjunction with Invacare® accessories. Accessories designed by other manufacturers have not been tested by Invacare® and are not recommended for use with Invacare® products.

WARNING!

– Skin condition must be checked very frequently after the installation of any new seating system. Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs. Working with your therapist, physician, and equipment supplier is the best way to ensure that a seating choice matches your individual needs. As the needs of the individual become more complex, the seating evaluation becomes more important.

WARNING!

Risk of severe injury when transporting the client within a wheelchair fitted with an Invacare® Matrx® Mini back in a motor vehicle.

– Wherever practicable Invacare recommends that the client is transferred out of the wheelchair into a vehicle seat using a lap and diagonal seat belt or an appropriate child safety seat.

– Invacare® Matrx® Mini backs may only be used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176-19.

– For safe transportation of a client within a wheelchair in a motor vehicle, refer to the user manuals of the wheelchair and the tie-down and occupant restraint system (WTORS).

3 Setup and usage

3.1 Prior to installation

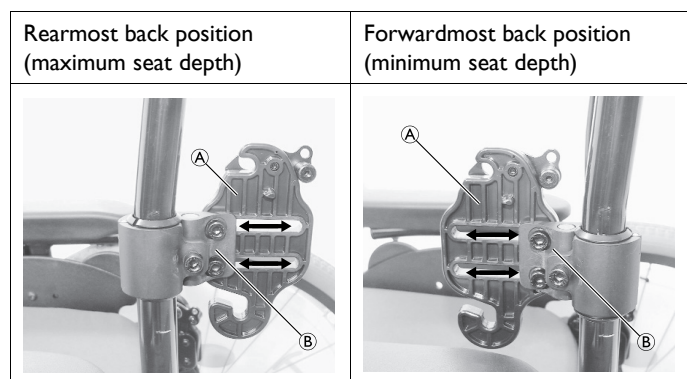
WARNING!

When establishing the mounting position for the Invacare® Matrx® Mini back, ensure the stability of the wheelchair is not compromised. Adjusting the forward and aft position of the backrest will alter the user's center of gravity within the wheelchair. A recessed back position can significantly reduce the rearward stability of the wheelchair. Similarly, a more forward back position will reduce the wheelchair's forward stability.

WARNING!

- The procedures in this manual should be performed by a qualified technician.
- After any adjustments, repair or service and before use, make sure that all attaching component parts are secure.
- Do not install the Invacare® Matrx® Mini assembly onto back canes with an outside diameter greater than 25 mm or less than 19 mm. Otherwise, injury or damage may occur.
- The mounting position of the Invacare® Matrx® Mini is directly related to the chair's stability. When the Invacare® Matrx® Mini is added to a tilt and/or reclining chair, it may cause a decrease in the chair's stability. It may be necessary to reposition the front casters, rear wheels, backrest angle, tilt-in-space, recline position and/or seat depth before use.
- Use extreme caution when using a new seating position.
- When a Matrx® Mini is added to a tilt and/or reclining chair, a rigidizer bar or crossbar must be present to stabilize the back canes.
- Ensure the Invacare® Matrx® Mini is properly secured to the wheelchair before using. Otherwise injury or damage may occur.

Re-configuring the mounting plate configuration

The mounting plates (A) are pre-assembled on the cane clamps (B) in the rearmost mounting configuration. This maximizes the seat depth of the wheelchair seat up to a maximum 64 mm rearward offset.

By changing the orientation of the mounting plates on the cane clamps, the assembly can be re-configured to provide a more forward mounting position for the back if desired up to a maximum 64 mm forward offset.

Each mounting plate is also designed to provide 38 mm of fore and aft adjustment relative to the cane clamp.

1. Detach the mounting plate from the cane clamp by removing the 2 depth/recline adjustment screws.
2. Remove the cane clamp from the backrest tube by removing the cane clamp screw.
3. Rotate the cane clamp 180° and re-fasten it to the mounting plate.
4. Install the cane clamp to the backrest tube as illustrated above.

Inserting plastic inserts (Fig. 2)

Accommodating different sized backrest tubes

Backrest tube diameter	Plastic inserts
25 mm	No plastic inserts
22 mm	22 mm plastic inserts
19 mm	19 mm plastic inserts

WARNING!

Risk of injury

- If there is an uneven seam along the backrest tubes where a smaller diameter tube is inserted into a larger diameter tube, the cane clamps must be mounted above or below the seam. Do not overlap.

1. Apply the inner cane clamp (C) to the backrest tube (D). Optionally position the plastic inserts (A) around the backrest tubes.
2. Close the outer cane clamp (B) around the backrest tube.

IMPORTANT!

Risk of damage

- When using the plastic inserts (A), make sure that they are correctly positioned between the cane clamp.

3. Fix the cane clamp assembly with screw (E).

3.2 Installing the backrest tube clamps (Fig. 3)

WARNING!

Risk of injury

- The installation hardware provided is high strength and tested for durability.
- Do not substitute hardware. Use only the hardware supplied.

1. Remove existing wheelchair back (if applicable).
2. Establish the desired mounting plate set-up/configuration for the user, see chapter 3.1 Prior to installation, page 5.
3. Loosen the cane clamp screw and loosely install the clamp (with mounting bracket) onto each backrest tube. (Install plastic inserts if required).
4. Starting on one side, position the backrest tube clamp at the desired height on the backrest tube and tighten the clamp screw enough to hold it in place. (The clamps can be fully tightened after the back is installed and all final adjustments have been made.)
5. From a fixed point on the wheelchair (e.g. seat frame/rigidizer bar) measure the exact height of the installed backrest tube clamp assembly, and install the second backrest tube clamp at the same height on the opposite backrest tube.

IMPORTANT!

- Make sure that the backrest tube clamps/mounting plates are mounted parallel with each other.

3.3 Installing the mounting pins (Fig. 4)

1. Insert the backing plate (A) in the correct orientation into the backrest pan (C).
2. Install the mounting pins (B).

3.4 Installing the Matrx® Mini backrest (Fig. 5)

1. Slide the lower mounting pins (B) into the hooked slots at the bottom of each mounting plate (C).
2. With the lower pins in place, lock/snap the upper mounting pins (A) into the latch mechanism at the top of the mounting plates.

WARNING!

- Always ensure both latch mechanisms are fully engaged around the upper mounting pins following installation.

Before proceeding with the final set-up, the mounting pins may require adjustment to ensure the backrest is properly fitted/secured to the wheelchair.

Due to variances that exist between different wheelchair manufacturers, the mounting pin depth may require adjustment to properly fit the Matrx® Mini back onto the wheelchair backrest tubes. Two sets of depth adjustable pins are supplied with every back: Standard Pins and Long Pins. Standard pins come factory installed and are suitable for most typical wheelchair configurations. Long pins are designed to allow installation onto back canes that are up to 25 mm wider than the nominal width of the backrest.

Optionally, an extended mounting pin (EMP) kit is available and allows a back to be installed on backrest tubes up to 51 mm wider than the back width.

WARNING! **Risk of injury**

- EMP kits are designed only for use with Matrx® Mini backs and standard sized Matrx® Elite backs (up to max. 136 kg weight capacity).

3.5 Adjusting the mounting pins (Fig. 6/7)

1. With the back secured in place on the wheelchair, determine the amount of adjustment required for each mounting pin (A).
2. Remove the backrest (D) from the wheelchair and use the wrench provided to loosen each flange nut (B). (If necessary, replace the standard mounting pins with long mounting pins (E)).
3. Adjust the mounting pin depth (clockwise to shorten/ counter-clockwise to lengthen) so that a minimum gap of 3 mm exists between the top of the pin and the mounting plate (C).
4. Re-tighten each flange nut to secure the pins in place. (Make sure that the pins do not rotate when tightening the flange nuts).

WARNING! **Risk of injury and damage**

- Improperly secured flange nuts may cause damage/cracking in the back pan and lead to accidents.
- Make sure that all flange nuts are fully tightened

WARNING! **Risk of injury and damage**

- Adjacent plastic components may degrade and become brittle (break) if contaminated.
- Never use Loctite® on mounting hardware.

WARNING! **Risk of injury and damage**

- Improper pin set-up may cause damage to the back pan and/or the mounting plates.
- Ensure sufficient spacing is provided between the mounting plates and the mounting pins, a minimum gap of 3 mm is recommended.

5. Verify that the overall fit and alignment of the back is correct (make further adjustments as needed).
6. Fully secure the backrest tube clamp assembly onto both back posts, and ensure the mounting plates do not slip or rotate.

3.6 Setting depth and angle of the backrest (Fig. 8)

1. The user must be seated in the wheelchair.
2. Loosen the depth/angle adjustment screws (A) on the mounting plates (B).
3. Adjust the back to the desired depth and recline angle.
4. Re-tighten all screws to lock the back in position.

Ensure that the depth/angle adjustment for the back is made symmetrically on the left and right mounting plates. The vertical lines (C) on the mounting plates are designed as a point of reference to assist with symmetrical mounting. The back position can be angle adjusted/rotated, so that the lines may be used to match the angle on both mounting plates.

3.7 Removing the Matrx® Mini back (Fig. 9)

1. Simultaneously lift the release levers (left & right) to unlock the latch mechanism (D) and release the upper pins (A) from the mounting plates (C).
2. Lift the entire backrest up and forward to disengage the lower pins (B) from the lower slots in the mounting plates.

For backrests with one-hand cord release, refer to chapter 3.8 Installing the one-hand cord release (optional) (Fig. 10), page 6.

3.8 Installing the one-hand cord release (optional) (Fig. 10)

1. Thread the Nylon cord (A) through the holes (B) in the latch mechanism at the top of the left and right mounting plates.
2. Install the spring loaded cord-Lock mechanism (C) onto the Nylon cord.
3. Press the button (D) to open the lock mechanism and slide the two open ends of the cord through the opening.
4. With the back properly installed on the wheelchair, adjust the cord length so that it hangs at approximately a 45° angle along the back pan (this will allow the release mechanism to operate more effectively).
5. Cut/remove excess cord to eliminate any risk of entanglement.
6. Combine the loose cord ends into a single knot (E) to prevent the cord-lock from slipping.
7. Carefully melt the ends (F) of the cord to prevent fraying.
8. Lift cord to release the back cushion (G).

The cord release mechanism is illustrated using a Standard Elite Back.

3.9 Installing/adjusting the lumbar support (optional) (Fig. 11)

The lumbar support (A) is a pre-fabricated foam insert that provides lower back support for additional comfort. The position of the lumbar insert can be adjusted inside the Matrx® Mini cover, or it may be removed if no lumbar support is desired.

1. Open outer cover at the bottom rear of the back pan (via Velcro® strip).

The lumbar foam may be cut/modified as needed to meet the needs of the end user.

2. If necessary, cut the lumbar foam to the appropriate size and insert the lumbar support inside the cover between the foam back cushion (B) and the back pan (C).
3. Adjust the lumbar support manually to any desired height/position/orientation, or remove completely if no lumbar support is required.
4. Refasten the cover (via Velcro® strip) onto the back pan.

4 Cleaning and care

IMPORTANT!

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- For further information on decontamination in health care environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CGI39 and your local infection control policy.

IMPORTANT!

Risk of product damage.

- Do not use cleaning agents containing bleach, phenols or alcohols.

Cleaning the cover

1. Wipe clean the outer cover with a damp cloth.
or
2. Carefully remove the cover by releasing the Velcro® hook and loop fasteners along the top and bottom edge and inside of the back shell.
3. Machine wash at 60° C using the delicate cycle (Instructions on label).

! **IMPORTANT!**
Washing at higher temperatures will cause shrinkage.

Drying the cover

1. Hang cover from a line or bar and drip dry in a clean indoor environment.
or
Tumble dry on a low heat setting.

! **IMPORTANT!**
– Tumble dry setting must not exceed 40 °C.
– Do not tumble dry for longer than 10 minutes.
– Dry thoroughly before re-fitting to the foam.

5 Back sizes/ weight capacity

Width	Height				Weight capacity
	180 mm (7")	260 mm (10")	300 mm (12")	360 mm (14")	
260 mm (10")	MINI1007	MINI1010	MINI1012	—	68 kg
280 mm (11")	MINI1107	MINI1110	MINI1112	—	
310 mm (12")	MINI1207	MINI1210	MINI1212	MINI1214	
330 mm (13")	MINI1307	MINI1310	MINI1312	MINI1314	
360 mm (14")	MINI1407	MINI1410	MINI1412	MINI1414	

6 Materials

Assembly hardware	Aluminum / Steel
Cover	100% Polyester Mesh with Clear Urethane laminate backing
Foam	2080 RBR Polyurethane

All materials used are corrosion resistant and not made with natural rubber latex.

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1. Bringen Sie die innere Klemme für das Gestänge © am Rückenlehnenrohr ⑩ an. Optional: Bringen Sie die Kunststoffeinsätze ① an den Rückenlehnenrohren an.
Invacare Limited
Invacare International GmbH
Advena Ltd.