


E2BKB - E2 BACKBONE FOR Matrix E2 BACKS

The **E2 BackBone (E2BKB) Support** is an optional contoured anatomical PSIS/Sacrolumbar support designed for use with the Matrix E2 Back Series (E2S and E2D Models).

The molded foam support is installed behind the E2 Back Cushion and secured directly onto the E2 back shell via hook and loop fastener. The size and position of the E2BKB Support may be modified/adjusted as necessary to meet the needs of the end user.

E2 BACKBONE INSTALLATION/ADJUSTMENT:

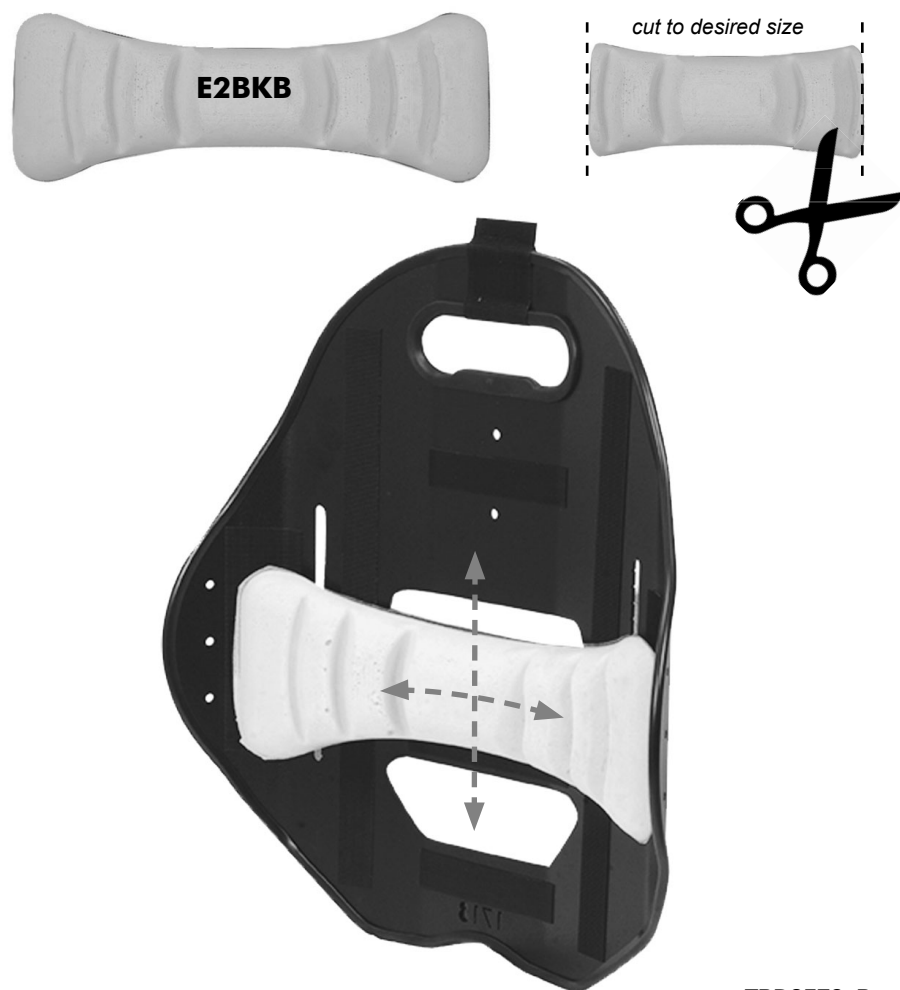

TRD0579 Rev B



IMPORTER:

France:
Invacare France Operations SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com; www.invacare.fr

United Kingdom:
Invacare Limited
Pencoed Technology Park,
Pencoed Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com; www.invacare.co.uk



DISTRIBUTOR:

Belgium & Luxembourg:
Invacare nv
Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
marketingbelgium@invacare.com
www.invacare.be

España:
Invacare SA
Avenida del Oeste, 50-50 - 1º-1a
Tel: (34) 972 493 214
contactsp@invacare.com
www.invacare.es

Danmark:
Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
denmark@invacare.com;
www.invacare.dk

Deutschland:
Invacare GmbH
Am Achener Hof 8
D-88316 Isny
Tel: (49) (0)7562 700 0
kontakt@invacare.com;
www.invacare.de

France:
Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com;
www.invacare.fr

Ireland:
Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords,
County Dublin
Tel: (353) 1 810 7084
ireland@invacare.com;
www.invacare.ie

Italia:
Invacare Mecc San s.r.l.,
Via Marco Corner, 19,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
italia@invacare.com; www.invacare.it

Nederland:
Invacare BV
Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
nederland@invacare.com
www.invacare.nl

Norge:
Besøksadresse: (Office Address)
Invacare AS
Brynsveien 16
0667 Oslo
Tel: (47) (0)22 57 95 00
norway@invacare.com;
www.invacare.no

Österreich:
Invacare Austria GmbH
Herzog Odilo-Strasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
info-austria@invacare.com;
www.invacare.at

Portugal:
Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
portugal@invacare.com
www.invacare.pt

Schweiz / Suisse / Svizzera:
Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
switzerland@invacare.com
www.invacare.ch

Sverige:
Invacare AB
Fagerstagatan 9
S-163 53 Spånga
Tel: (46) (0)8 761 70 90
sweden@invacare.com;
www.invacare.se

Suomi:
Camp Mobility
Patamäenkatu 5,
33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi


United Kingdom:
Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com;
www.invacare.co.uk

USA:
Invacare America
Elyria, Ohio,
USA 44036-2125
Tel: (1) 800 333 6900
www.invacareamerica.com

Motion Concepts USA
700 Ensminger Rd. Suite 112
Tonawanda, NY 14150
Tel: (1) 888.433.6818
www.motionconcepts.com



Motion Concepts
505 Cityview Blvd, Unit 1
Vaughan, Ontario, Canada L4H 0L8
Tol Free Tel: (1) 866.748.7943
Tel: (1) 905.695.0134
www.motionconcepts.com

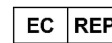

Advena Ltd.
Tower Business Center, 2nd Fl.,
Tower Street, Swatar, BKR 4013 Malta




E2BKB - E2 BACKBONE FOR Matrix E2 BACKS

The **E2 BackBone (E2BKB) Support** is an optional contoured anatomical PSIS/Sacrolumbar support designed for use with the Matrix E2 Back Series (E2S and E2D Models).

The molded foam support is installed behind the E2 Back Cushion and secured directly onto the E2 back shell via hook and loop fastener. The size and position of the E2BKB Support may be modified/adjusted as necessary to meet the needs of the end user.

E2 BACKBONE INSTALLATION/ADJUSTMENT:

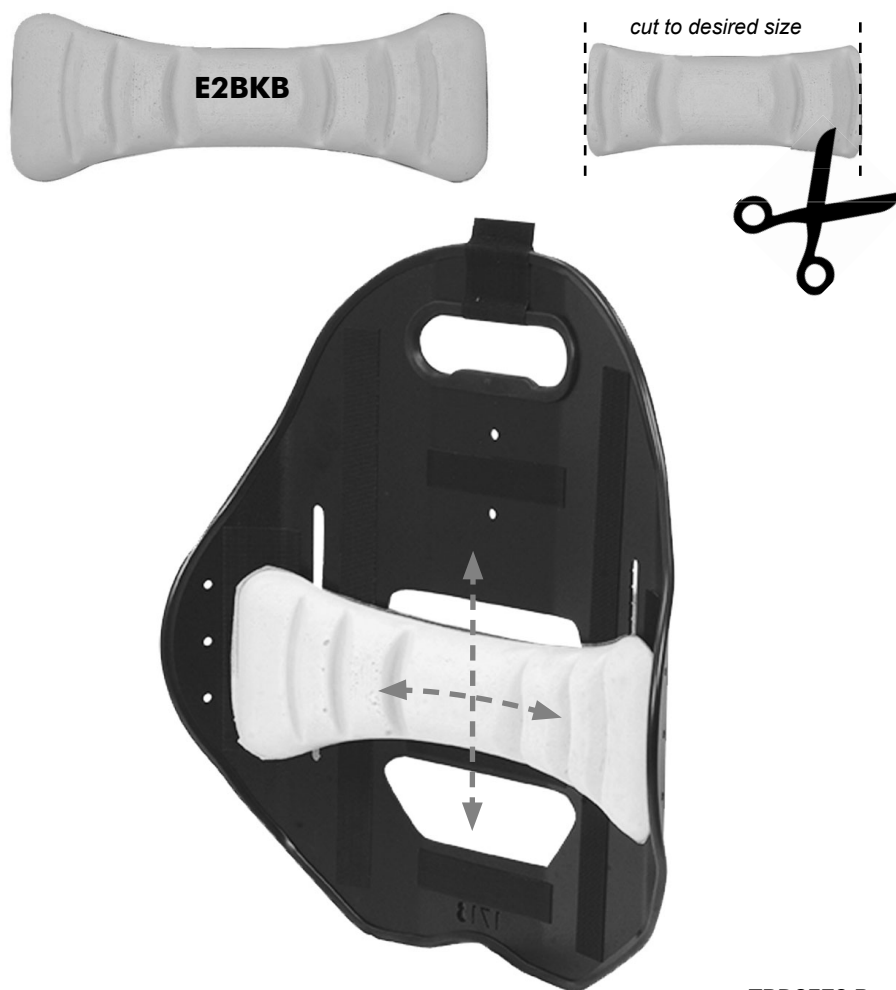

TRD0579 Rev B



IMPORTER:

France:
Invacare France Operations SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com; www.invacare.fr

United Kingdom:
Invacare Limited
Pencoed Technology Park,
Pencoed Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com; www.invacare.co.uk



DISTRIBUTOR:

Belgium & Luxembourg:
Invacare nv
Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
marketingbelgium@invacare.com
www.invacare.be

España:
Invacare SA
Avenida del Oeste, 50-50 - 1º-1a
Tel: (34) 972 493 214
contactsp@invacare.com
www.invacare.es

Denmark:
Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
denmark@invacare.com;
www.invacare.dk

Deutschland:
Invacare GmbH
Am Achener Hof 8
D-88316 Isny
Tel: (49) (0)7562 700 0
kontakt@invacare.com;
www.invacare.de

France:
Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com;
www.invacare.fr

Ireland:
Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords,
County Dublin
Tel: (353) 1 810 7084
ireland@invacare.com;
www.invacare.ie

Italia:
Invacare Mecc San s.r.l.,
Via Marco Corner, 19,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
italia@invacare.com; www.invacare.it

Nederland:
Invacare BV
Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
nederland@invacare.com
www.invacare.nl

Norge:
Besøksadresse: (Office Address)
Invacare AS
Brynsveien 16
0667 Oslo
Tel: (47) (0)22 57 95 00
norway@invacare.com;
www.invacare.no

Österreich:
Invacare Austria GmbH
Herzog Odilo-Strasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
info-austria@invacare.com;
www.invacare.at

Portugal:
Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
portugal@invacare.com
www.invacare.pt

Schweiz / Suisse / Svizzera:
Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
switzerland@invacare.com
www.invacare.ch

Sverige:
Invacare AB
Fagerstagatan 9
S-163 53 Spånga
Tel: (46) (0)8 761 70 90
sweden@invacare.com;
www.invacare.se

Suomi:
Camp Mobility
Patamäenkatu 5,
33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi


United Kingdom:
Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 200
uk@invacare.com;
www.invacare.co.uk

USA:
Invacare America
Elyria, Ohio,
USA 44036-2125
Tel: (1) 800 333 6900
www.invacareamerica.com

Motion Concepts USA
700 Ensminger Rd. Suite 112
Tonawanda, NY 14150
Tel: (1) 888.433.6818
www.motionconcepts.com



Motion Concepts
505 Cityview Blvd, Unit 1
Vaughan, Ontario, Canada L4H 0L8
Tol Free Tel: (1) 866.748.7943
Tel: (1) 905.695.0134
www.motionconcepts.com

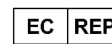

Advena Ltd.
Tower Business Center, 2nd Fl.,
Tower Street, Swatar, BKR 4013 Malta

