

## PROTE2S/PROTE2D MATRX PROTECT COVER

Our 'Standard' and 'Deep' Protect Covers provide a removable, wipeable and machine washable infection control slipcover for use with Matrix Elite and E2 Backs.

**STANDARD COVERS = PROTE2Swwhh; DEEP COVERS = PROTE2Dwwhh**

Our Matrix Protect Covers are designed with hermetically welded seams to prevent entrapment of particles and passage of fluids, and compatible with hospital grade disinfectants.

### MATRX PROTECT COVER INSTALLATION/REMOVAL:



### LAUNDERING THE MATRX PROTECT COVER:

**i IMPORTANT!** The Protect Cover MUST be removed from the Matrix Back Support PRIOR to laundering. (**Note:** Do Not Dry Clean, Do Not Bleach).

**i IMPORTANT!** Adequate drying is a crucial factor in the care and maintenance of the covers. The covers should be dried completely prior to use on the cushion. Air drying of covers is recommended.

#### Machine Wash or Hand-Wash:

- Remove the Protect Cover from the Matrix Back Support.
- The Protect Cover can be hand washed or machine-washed in warm water not higher than 60°C (140°F), on a delicate cycle. with neutral detergents such as Tide or Persil.
- To avoid possible damage, ensure the cover does not come in contact with any other sharp garment accessories during laundering.

## PROTE2S/PROTE2D MATRX PROTECT COVER

### Drying:

- DO NOT WRING OUT fabric. Hang to dry on a towel-covered wood hanger and drip dry to minimize wrinkling.
- Alternatively, covers may be tumble-dried on a low/cool setting. Cushion covers should not be ironed or pressed.

*ii Polyurethane (PU) coated fabrics can absorb liquids for short periods causing a temporary change to the PU characteristics. The wet cover is more vulnerable to physical damage and should not be used until fully dried.*

### CLEANING AND DISINFECTION INSTRUCTIONS:

The coated side of the **Matrix Protect Cover** can be cleaned and disinfected with ready-to-use disinfecting wipes 3M, Virox Accel TB, Accel Prevention, PREempt, OPTIM 33 TB, or other wipes with similar formulations. Note that the surface should be cleaned first, with all gross contamination (i.e. urine, feces, blood) first removed with absorbent material (cloth or paper towel). Ensure the coated surface of the cover is clean by wiping with one of 2-in-1 cleaning and disinfecting wipes mentioned above.

To disinfect, liberally wipe the surface with the ready-to-use disinfecting wipe again. Avoid pooling of the disinfectant and ensure the disinfectant is distributed equally over the cover surface. Wipe dry after recommended dwell time (refer to each disinfecting products manufacturing instructions). It is best practice to wipe the surface with a damp cloth, mild soap, and finally a damp cloth no matter what disinfectant is used.



#### Motion Concepts

**USA**  
700 Ensminger Rd. Suite 112  
Tonawanda, NY 14150  
888-433-6818

#### www.motionconcepts.com

**Canada**  
84 Citation Dr., Unit 1-10  
Concord, Ontario L4K 3C1  
905-695-0134  
866-748-7943

Copyright © 2022 Motion Concepts LP. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Motion Concepts. Trademarks are identified by <sup>TM</sup> and <sup>®</sup>. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

**TRD0592 Rev B**