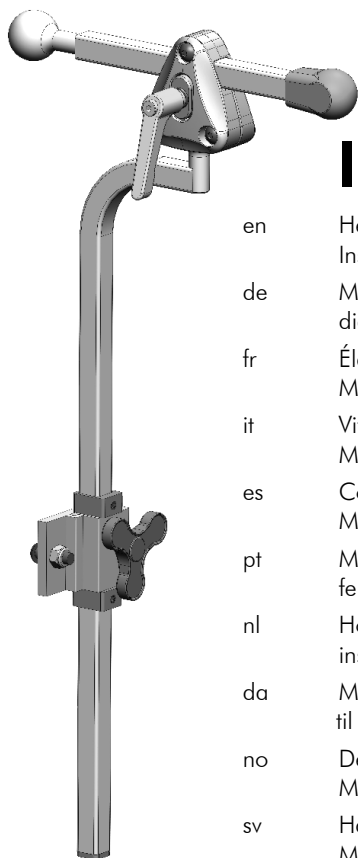
**PATENT
PENDING**

matrix®

LOXX™

Invacare® Matrix® LOXX™

en	Headrest Hardware Installation Manual	6
de	Montagehandbuch für die Kopfstützenbefestigung	8
fr	Éléments d'appui-tête Manuel d'installation	10
it	Viteria per il poggiatesta Manuale d'installazione	12
es	Componentes del reposacabezas Manual de instalación	14
pt	Manual de instalação das ferragens do apoio de cabeça	16
nl	Hoofdsteenmontagesysteem installatiehandleiding	18
da	Monteringsbeslag til hovedstøtte	20
no	Deler for hovedstøtte Monteringsveiledning	22
sv	Hårdvara för huvudstöd Monteringsanvisningar	24
fi	Päätuen tarvikkeiden asennusopas	26

matrix®
SEATING SERIES

TRD0668 Rev A

en This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.

This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

DEALER: This manual **MUST** be given to the user of the product.

USER: Before using this product, read this manual and save for future reference.

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i IMPORTANT! As applicable, please refer to the OEM Headrest Pad Owners Manual (*provided separately*) for additional safety information, warnings and cautions related to the installation and care of the headrest pad.

2.1 SAFETY SYMBOLS:

The following symbols are used throughout this user manual to identify warnings, cautions and important notes. It is very important that you read and understand them completely.

i IMPORTANT! Indicates important information which if not followed could result in damage to the product.

! CAUTION! Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

! WARNING! Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

Manufacturer

CE This product complies with the EU Medical Device Regulation 2017/745 for medical products (Class I).

2.2 WARRANTY

We provide a manufacturers warranty for this product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the device was purchased.

2.3 SERVICE LIFE

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary depending on frequency and intensity of use.

3.0 LOXX™ HEADREST HARDWARE INSTALLATION

The LOXX™ Headrest Mounting Hardware is designed for use with most industry standard headrest pads, and is fully compatible with our line of Matrx Elan Headrest Pads (sold separately).

! WARNING! Risk of Serious Injury
Injury may occur during use of the wheelchair if a headrest is wrongly adjusted or not installed correctly. The users neck may become hyperextended in the event of a collision, or when the system is tilted or reclined.
• *This product must be fitted by an experienced clinician or technician.*
• *Before each use, check that all mounting/adjustment hardware is securely fastened.*

! WARNING! Risk of Serious Injury
Risk of serious injury when transporting a client in a motor vehicle within a wheelchair fitted with the Invacare® Matrx® LOXX™ headrest hardware.
• *Wherever practicable Motion Concepts recommends that the client is transferred out of the wheelchair into a vehicle seat using a lap and diagonal seat belt or an appropriate child safety seat.*
• *The Matrx® LOXX™ Hardware may only be used in conjunction with wheelchairs that comply with the performance requirements of RESNA WC19 or ISO 7176–19.*
• *For safe transportation of a client within a wheelchair in a motor vehicle, refer to the user manuals of the wheelchair and the tie-down and occupant restraint system (WTORS).*

3.0 LOXX™ HEADREST HARDWARE INSTALLATION

WARNING! Risk of injury

- Inspect the headrest hardware to ensure that there are no missing or damaged parts.

WARNING! Risk of injury

- Before drilling or modifying any back shell, always contact the back manufacturer to ensure the back is recommended for a headrest application.

3.1 LOXX™ INSTALLATION/SET-UP INSTRUCTIONS

(Refer to images provided in **Section 1.0 - ASSEMBLY OVERVIEW** (p.4-5))

OPTIONAL LOCKING SCREW (see **Fig 1**): The locking lever (**A1**) may be replaced with the optional 30mm locking screw (**A2**) provided. Unscrew the lever from the upper clamp housing and retain the original washer (a). Being careful to keep the clamping block (**b**) in position inside the upper pivot, use the 5mm hex key (provided) to install the replacement locking screw (with washer).

MOUNTING CLAMP INSTALLATION (see **Fig 2**):

Secure the mounting clamp assembly (**E**) onto back shell* using hardware provided. If additional clearance is required, install the spacer plate and long screws (**c**) provided. (**note: the mounting clamp hardware is installed from the inside of the back shell*)

LOXX™ HARDWARE INSTALLATION (see **Fig 3**):

- Loosen set-screw (**d**) and remove the lower D-Ring (**D2**) from the headrest post (**1**).
- Install the headrest post (**F**) into the mounting clamp assembly (**E**), and adjust the headrest post to the desired height/position (**2**). (*note: loosen upper D-Ring if necessary*)
- Tighten the mounting clamp (**E**) to secure the headrest post in position (**3**).
- Adjust the upper D-Ring (**D1**) so that it rests flush with the top of the mounting clamp (**4**), then secure the D-Ring into place via the set-screw (**d**).
- Reinstall and secure the Lower D-Ring (**D2**) onto the headrest post (**5**) so that it rests flush with the bottom of the mounting clamp assembly.

HEADREST POST MODIFICATIONS:

If necessary, the headrest post (**F**) can be cut (shortened) to prevent any collision interference along the back of the wheelchair/seating system. (*Note: After cutting the post, the plastic end cap (**G**) can be removed and re-installed into the newly cut post.*)

HEADREST PAD ADJUSTMENT (see **Fig 4**):

Loosen the locking lever (**A1**) or locking screw (**A2**) on the upper pivot assembly (**B**). Slide the horizontal bar (**C**) to the desired depth inside the upper pivot (**1**), and adjust the headrest angle by pivoting/rotating the upper pivot to achieve the desired headrest position (**2**). Re-tighten the locking lever/locking screw (**B**) to secure the headrest into position.

ELAN HEADREST PAD* INSTALLATION (see **Fig 5**):

Position the headrest mounting collar over the pivot ball on the horizontal bar (**1**); then align and secure the mounting collar to the headrest pad (**2**).

**Note: Headrest pad/mounting hardware sold separately.*

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Advena Ltd.